


<b>Brand Name of Product</b>	AquaTest™
<b>Generic Name of Product</b>	Water Test
<b>Product Code Number(s)</b>	AT101
<b>Purpose of Product</b>	To test water quality critical to proper cleaning, including pH, hardness, and alkalinity.
<b>Range of Applications for Product</b>	Water used in the cleaning process
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>3- in-1 water quality test strips. <ul style="list-style-type: none"> <li>pH (2–12)</li> <li>Alkalinity (0–500ppm CaCO<sub>3</sub>)</li> <li>Hardness (0–1000 ppm CaCO<sub>3</sub>)</li> </ul> </li> <li>Result within 30 seconds</li> <li>Easy to interpret colorimetric results.</li> </ul>

Shipping & Storage	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	Eighteen (18) months from date of manufacture.

Instructions for Using Product																									
Description of Use (s)	To test water quality used in the cleaning.																								
Preparation for Use	N/A																								
Diagrams (drawings, pictures)																									
	<table><tr><th>Water Quality Measurement</th><th>Units</th><th>Utility Water</th><th>Critical Water</th><th>Steam</th></tr><tr><td>pH @ 25 °C</td><td>pH</td><td>6.5–9.5</td><td>5.0–9.5</td><td>5.0–9.2</td></tr><tr><td>Total Alkalinity</td><td>MgCaCO<sub>3</sub>/L</td><td>&lt; 400</td><td>&lt; 8</td><td>&lt; 8</td></tr><tr><td>Total Hardness</td><td>MgCaCO<sub>3</sub>/L</td><td>&lt; 150</td><td>&lt; 1</td><td>&lt; 1</td></tr></table>					Water Quality Measurement	Units	Utility Water	Critical Water	Steam	pH @ 25 °C	pH	6.5–9.5	5.0–9.5	5.0–9.2	Total Alkalinity	MgCaCO <sub>3</sub> /L	< 400	< 8	< 8	Total Hardness	MgCaCO <sub>3</sub> /L	< 150	< 1	< 1
	Water Quality Measurement	Units	Utility Water	Critical Water	Steam																				
	pH @ 25 °C	pH	6.5–9.5	5.0–9.5	5.0–9.2																				
	Total Alkalinity	MgCaCO <sub>3</sub> /L	< 400	< 8	< 8																				
	Total Hardness	MgCaCO <sub>3</sub> /L	< 150	< 1	< 1																				
Steps for Use of Product	<ul style="list-style-type: none"><li>Collect water from the cold-water/hot-water source into a clean container:<ol style="list-style-type: none"><li>Dip the entire test strip into water for 5 seconds, then remove.</li><li>Shake once briskly to remove excess water from strip.</li><li>Wait 20 seconds.</li><li>Compare each pad to the corresponding color chart to determine the value for pH, alkalinity, and hardness. (Note: Refer to the above modified table for values from ANSI/AAMI ST108 Water quality Monitoring- requirements.)</li><li>Repeat steps 1–4 for hot-water source.</li></ol></li></ul>																								
Interpretation of Results	<ol style="list-style-type: none"><li>Color for each pad corresponds to the value indicated on the chart.</li><li>Record results on the Log Sheet.</li><li>Report any deviations from expected values.</li></ol>																								
Contraindications of Test Results	N/A																								
Documentation	N/A																								
Special Warnings and Cautions	⚠ CAUTION: HOT WATER. Be careful not to get burned while performing test (based on how hot your facility’s water is).																								
Disposal	N/A																								

Reprocessing Instructions	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A

<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	N/A
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A

<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	N/A
<b>Reference Documents</b>	ProFormance™ Monitoring Products Brochure, ProFormance™ Price List
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>