



Instructions for Use: Universal Container Card

Brand Name of Product	SteriTec Universal Container Card
Generic Name of Product	Process Indicator
Product Code Number(s)	CI-138
Purpose of Product	The SteriTec Universal Container Card is a steam sterilization process indicator designed for use with various types of sterilization containers. The Container Cards will function in sterilizers that operate at 121°C to 134°C (250°F to 273°F). It provided a visible confirmation that the instrument container being sterilized has been exposed to steam sterilization processing.
Range of Applications for Product	
Key Specifications of Product	<ul style="list-style-type: none"> • When exposed to a steam sterilization process, the indicator bar changes color, thus verifying that critical parameters were met • Color change: White to black • Complete color change: <ul style="list-style-type: none"> ○ 10 minutes ± 30 seconds at 121°C ○ 2 minutes ± 30 seconds at 132-134°C • Indicator stability: Excellent before and after exposure • Physical characteristics: <ul style="list-style-type: none"> ○ The card can fit in various container card-holders by folding or not folding the top sizing flap to the appropriate size

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	Store at 10-38°C (50-100°F) and at 10-70% relative humidity in the re-sealable moisture barrier bag provided.
Packaging Contents	
Shelf Life	Please refer to the expiration date printed on the product packaging.

Instructions for Using Product	
Description of Use (s)	Designed to give visible indication that sterilizing conditions were met in a steam sterilizers that operate at 121°C to 134°C (250°F to 273°F).
Preparation for Use	Remove the desired number Universal Container Cards from the package and re-seal the bag.
Diagrams (drawings, pictures):	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Complete the appropriate information on the Container Card. 2. Slide the Container Card in each container card holder for sterilization. Adjust the card size by folding along the perforated top edge. 3. Sterilize container as usual in a steam sterilizer. 4. Following sterilization cycle, verify that indicator bar has turned from white to black. <p><i>If there is any doubt about sterility of an item, it must be considered NOT sterile.</i></p>
Interpretation of Results	Color change from white to black indicates that sterilizing conditions were met.
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	

Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	

Related Healthmark Products	Steam Sterilization Integrators, Steam Sterilization Monitors
Other Product Support Documents	Steriking™ Sterilization Products Brochure, Steriking™ Sterilization Products Price List
Reference Documents	SteriTec Product Technical Data Sheet
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com