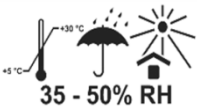
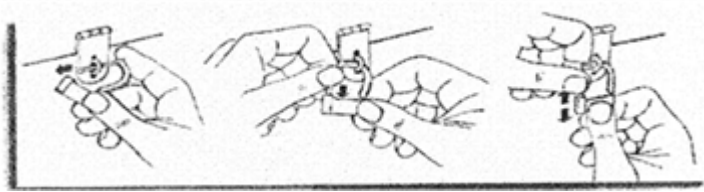
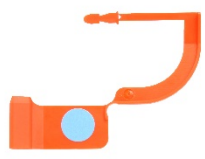


<b>Brand Name of Product</b>	Container Loks
<b>Generic Name of Product</b>	Rigid container tamper evident seal
<b>Product Code Number(s)</b>	3030 WT, 3030 WTPDTS
<b>Intended Use</b>	Use the tamper-evident seal to indicate the container has not been opened since sterilization.
<b>Range of Applications for Product</b>	N/A
<b>Key Specifications of Product</b>	VH2O2 process indicator dot

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	Product should be stored in closed cabinets or otherwise protected from any UV-light source. The process indicator is sensitive to UV-light and may change color over time if not stored properly. 
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	N/A

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	Use the tamper-evident seal to indicate the container has not been opened since sterilization.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	 
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Pass arrow shackle through opening of item to be sealed.</li> <li>2. Firmly push arrow shackle into locking device until it engages.</li> <li>3. Pull on seal to ensure it has been properly locked.</li> </ol>
<b>Interpretation of Test Results</b>	<i>VH2O2 sterilization:</i> If purchased with optional VH2O2 process indicator dot, reactive ink will turn blue in the presence of VH2O2.
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	N/A

<b>Special Warnings and Cautions</b>	If a seal is broken, assume sterility has been compromised and the tray must be reprocessed.
<b>Disposal</b>	N/A

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	<ul style="list-style-type: none"> <li>• Tamper-evident seal is materially compatible with steam (VH2O2/O3/EO sterilization).</li> <li>• If purchased with optional VH2O2 process indicator dot, reactive ink will change color in presence of VH2O2.</li> </ul>
<b>Storage</b>	Should be stored in closed cabinets or otherwise protected from any UV-light source. The process indicator is sensitive to UV-light and may change color over time, if not stored properly.
<b>Additional Information</b>	N/A
<b>Related Healthmark Products</b>	NA
<b>Other Product Support Documents</b>	Sterilization Packaging Products Brochure, Sterilization Packaging Products Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com