



Instructions for Use: Count Sheet Holders

| | |
|--|---|
| Brand Name of Product | Count Sheet Holders |
| Generic Name of Product | Count Sheet Holders |
| Product Code Number(s) | PBR-46, PBR-76 |
| Purpose of Product | Designed to hold instrument count sheets on the outside of wrapped trays (in compliance with AORN recommended practices). |
| Range of Applications for Product | For all trays with count sheets that accompany the set of surgical instruments. |
| Key Specifications of Product | <ul style="list-style-type: none"> • PBR-46: <ul style="list-style-type: none"> ○ Transparent poly front with medical grade paper back. ○ 4- x 6 inches. • PBR-76: <ul style="list-style-type: none"> ○ Medical grade paper back ○ 7 ½- x 3 inches. |

| Shipping & Storage | |
|---|-----|
| Shipping Conditions & Requirements | N/A |
| Storage Conditions | N/A |
| Packaging Contents | N/A |
| Shelf Life | N/A |

| Instructions for Using Product | |
|--|---|
| Description of Use(s) | Holds instrument count sheets on the outside of wrapped trays. |
| Preparation for Use | N/A |
| Diagrams (drawings, pictures): | N/A |
| Steps for Use of Product | <ol style="list-style-type: none"> 1. Fold the count sheet over to fit inside the medical grade paper pouch. 2. Peel away the protective layer to expose adhesive backing and adhere the medical grade paper pouch to the side of your wrapped package. |
| Interpretation of Results | N/A |
| Contraindications of Test Results | N/A |
| Documentation | Compliant with AORN guidelines, which recommend placing count sheets outside of sterilization trays. |
| Special Warnings and Cautions | N/A |
| Disposal | N/A |

| Reprocessing Instructions | |
|---|--|
| Point of Use | N/A |
| Preparation for Decontamination | N/A |
| Disassembly Instructions | N/A |
| Cleaning – Manual | N/A |
| Cleaning – Automated | N/A |
| Disinfection | N/A |
| Drying | N/A |
| Maintenance, Inspection, and Testing | N/A |
| Reassembly Instructions | N/A |
| Packaging | N/A |
| Sterilization | Materially compatible with steam and EO sterilization. |
| Storage | N/A |
| Additional Information | N/A |

| | |
|--|--|
| Related Healthmark Products | Slotted Tray Labels |
| Other Product Support Documents | Sterilization Packaging Price List, Sterilization Packaging Brochure |

| | |
|---------------------------------|--|
| Reference Documents | N/A |
| Customer Service Contact | Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com |