

Instructions for Use: Decontamination Gown

Brand Name of Product	Decontamination Gown
Generic Name of Product	Decontamination Gown
Product Code Number(s)	MN110 SMMD, MN110 LGXL, MN110 2X3X, MN110 4X
Intended Use	To keep clothing dry from splash up with a liquid impervious zone and shield from gross soil and bioburden in the decontamination area.
Range of Applications for Product	To help protect the wearer from contamination when working in a decontamination area.
Key Specifications of Product	<ul style="list-style-type: none"> ● Comprised of spun bonded polypropylene ● Upper and Lower zones are manufactured of a breathable material. ● Middle zone is manufactured of a liquid impervious zone covering the waist area. <ul style="list-style-type: none"> ○ Length is 11.75” inches across. ○ High level strikethrough resistance in the most vulnerable zone. ● Wrap-around gown (360° coverage) with elastic wrists ● Ties at neck and waist ● Thumb loop

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	



Instructions for Using Product	
Description of Use(s)	To help keep the wearer from becoming wet or contaminated in the decontamination area.
Preparation for Use	Remove gown from packaging.
Diagrams (drawings, pictures)	<p>Donning Gown 1st Time:</p> <div style="display: flex; justify-content: space-around;"> <div style="text-align: center;">  <p>Figure 1</p> </div> <div style="text-align: center;">  <p>Figure 2</p> </div> </div>



Figure 3



Figure 3



Figure 3



Figure 4



Figure 5



Figure 5



Figure 5



Figure 5



Figure 6

Doffing PPE gown 1st time:



Figure 1



Figure 2



Figure 2



Figure 2



Figure 2



Figure 2



Figure 3



Figure 4



Figure 4



Figure 5



Figure 5



Figure 5



Figure 6



Figure 7



Figure 8



Figure 9



Figure 10

Redonning new PPE gown 2nd time:



Figure 1



Figure 2



Figure 3



Figure 4



Figure 4



Figure 4



Figure 5



Figure 5

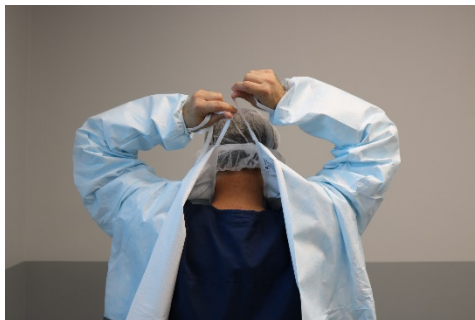


Figure 6



Figure 7

Steps for Use of Product

Donning PPE gown 1st time:

1. Put on hair covering. **Fig. 1**
2. Next put on mask (if applicable).
3. Then put on face shield. **Fig. 2**
4. Select size of gown to be worn.
5. Unfold and insert arms into the sleeves of the gown and slide thumbs through the thumb loops on the elastic wrists with the opening of the gown in the back. **Fig. 3**
6. Tie the neck ties behind the gown. **Fig. 4**
7. Take the ties at the waist to the back of the gown and tie in the back. (Do not bring the ties to the front of the gown and tie). **Fig. 5**
8. Put on shoe covers. **Fig. 6**
9. Put on gloves.

Doffing PPE gown 1st time:

1. First remove face shield. **Fig. 1**
2. Next remove gloves. **Fig. 2**

	<ol style="list-style-type: none"> 3. Then remove mask (if applicable) 4. Untie the neck ties behind the gown with bare hands. Fig 3 5. Untie the waist ties behind the gown with bare hands. Fig. 4 6. Slip out your thumb from the thumb lopes while holding and pulling away from you while slipping your hand through the gown. Fig. 5 7. With your hands in the sleeve of the gown, proceed to pull from the gown, then proceed to pull the front of the upper torso of the gown away from you. Fig. 6 8. With the clean side facing outward, remove your hands and arms from the gown. Fig. 7 9. Hang the gown with the contaminated side on the hook, without touching the outside of the gown with your bare hands so not to contaminate them. Fig. 8 10. Remove hair covering. Fig. 9 11. Then remove shoe covers. Fig. 10 <p>Redonning PPE gown 2nd time:</p> <ol style="list-style-type: none"> 1. Put on hair covering. Fig. 1 2. Next put on mask (if applicable). 3. Then put on face shield. Fig. 2 4. Put on shoe covers. Fig. 3 5. From the clean side of the gown, slip one arm into the gown and remove from the hook, and then slip the other arm and torso into the gown without touching the contaminated side. Fig. 4 6. Next, slip your thumbs through the thumb loops. Fig. 5 7. Tie the neck ties. Fig. 6 8. Put on gloves. 9. Then tie the waist ties behind the gown. (Do not bring the ties to the front of the gown and tie). Fig. 7
Interpretation of Test Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	The Decontam Gown is intended for use in the reprocessing area of healthcare facilities. It is not intended for use as an Isolation Gown or a Surgical Gown.
Disposal	One person-one day. Dispose of in accordance with facility policy.

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	
Related Healthmark Products	Disposable Jump Suit, Disposable Boot Legs, Face Shield
Other Product Support Documents	PPE Brochure, PPE Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com