

Instructions for Use: Disposable SST Tray

Brand Name of Product	Disposable SST Tray
Generic Name of Product	Disposable Tray
Product Code Number(s)	DSST-001-10
Intended Use	To safely transport contaminated items to the decontamination area.
Range of Applications for Product	Transporting contaminated items.
Key Specifications of Product	<ul style="list-style-type: none"> Recycled plastic that is 100% recyclable Lid: 11.771 inches W x 12.896 inches L Tray: 11.771 inches W x 12.896 inches L x 4 inches D

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

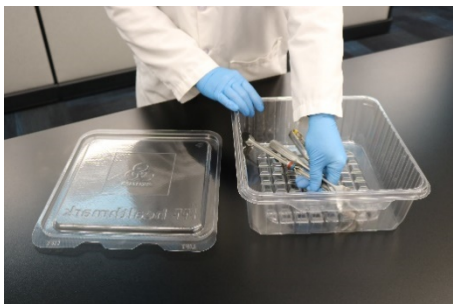

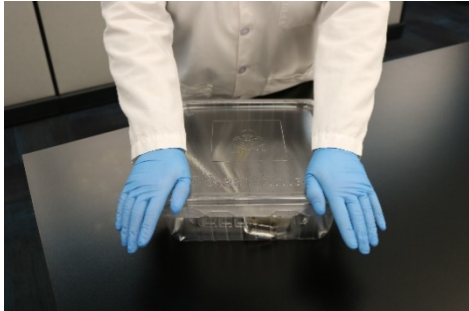
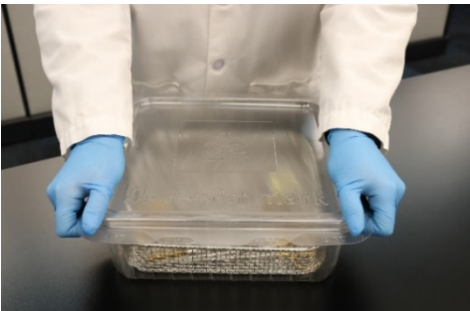
Instructions for Using Product	
Description of Use(s)	To safely transport contaminated items to the decontamination room.
Preparation for Use	N/A
Diagrams (drawings, pictures)	<div style="display: flex; flex-wrap: wrap;"> <div style="width: 50%; text-align: center;">  <p>Figure 1</p> </div> <div style="width: 50%; text-align: center;">  <p>Figure 1a</p> </div> <div style="width: 50%; text-align: center;">  <p>Figure 2</p> </div> <div style="width: 50%; text-align: center;">  <p>Figure 2a</p> </div> </div>



Figure 2b

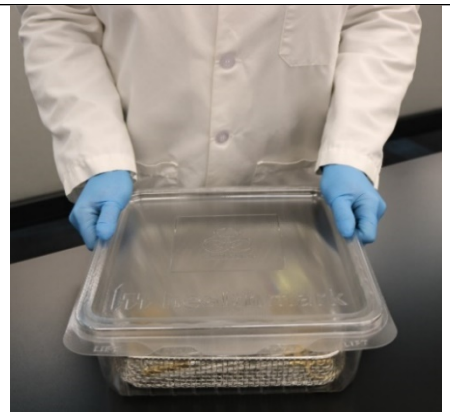


Figure 2c



Figure 3

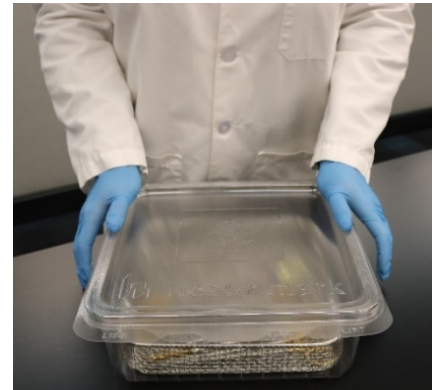


Figure 3a

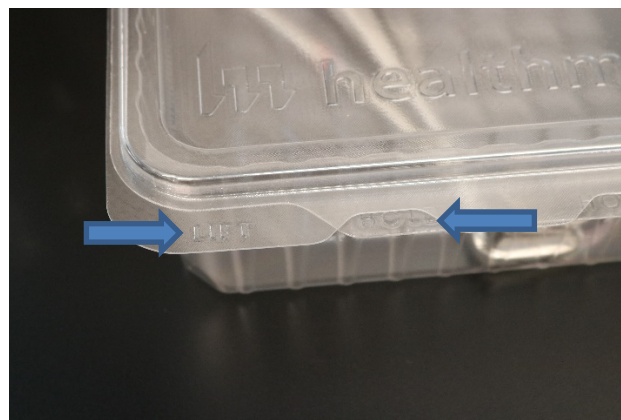


Figure 4

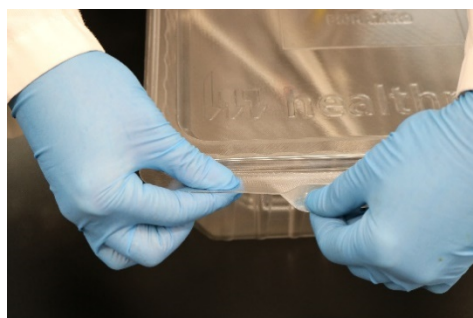


Figure 4a

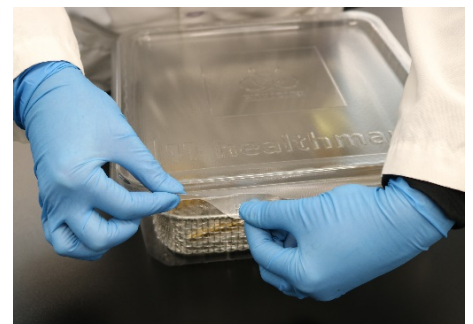





Figure 4b

	 <p>Figure 4c</p>  <p>Figure 5</p>  <p>Figure 5a</p>
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place contaminated item(s) in the tray. (Fig. 1, 1a). 2. Place lid onto the tray. <ol style="list-style-type: none"> a. Press down with your hands on the rim of the lid for a secure closure around the rim of the tray. (Fig. 2, 2a, 2b, 2c). b. Check all areas of the lid to ensure it is sealed prior to transportation. (Fig. 3, 3a). 3. To open the transport tray, place your thumb and index fingers of both hands on the “Lift” and “Hold” tabs shown on the lid and the tray. (Fig 4, 4a, 4b, 4c). 4. With your thumb and index finger, lift the lid up while your other thumb and index finger continue to hold the tray. (Fig. 5, 5a). <p>NOTE: All pictures are shown with and without a wire basket.</p>
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	Single use. Handle in compliance with facility policy for contaminated recyclable items.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	PPE Brochure, PPE Price List

Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com