



## Instructions for Use: Glove Box Dispenser-Single & Triple with Dividers

<b>Brand Name of Product</b>	Glove Box Dispenser
<b>Generic Name of Product</b>	Glove Box Dispenser
<b>Product Code Number(s)</b>	GBD-S1, GBD-T3
<b>Intended Use</b>	To hold one box or three boxes of extended-cuff gloves to be placed in all patient care areas requiring extended-cuff gloves.
<b>Range of Applications for Product</b>	Used in any area using hazardous chemicals: surgical areas, Sterile Processing areas,, oncology, pharmacy, and veterinary, or any area for the use of decontamination processes.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>● Holds one or three boxes of gloves</li> <li>● Made of clear PETG Plastic- Copolyester resin</li> <li>● Keyholes for wall mounting</li> <li>● Dimensions of glove box dispenser               <ul style="list-style-type: none"> <li>○ Single box dispenser                   <ul style="list-style-type: none"> <li>- 5.45” W x 16.10” H x 2.95” D (13.8 cm x 40.9 cm x 7.5 cm)</li> </ul> </li> <li>○ Weighs 0.8 lbs.</li> <li>○ Triple box dispenser with dividers                   <ul style="list-style-type: none"> <li>- 15.70” W x 16.10” H x 2.70” D (39.9 cm x 40.9 cm x 6.9 cm)</li> <li>- 1.8 lbs.</li> </ul> </li> </ul> </li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	To hold one box or three boxes of gloves to be placed in all patient care areas.
<b>Preparation for Use</b>	<ol style="list-style-type: none"> <li>1. Remove the dispenser from packaging.</li> <li>2. Securely mount to the wall by inserting the screws into the wall.</li> <li>3. Place the dispenser onto the screws at the bottom.</li> <li>4. Then place the dispenser with the two screw holders that are located on the back of the dispenser onto the screws to hold the dispenser in place.</li> </ol>
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ul style="list-style-type: none"> <li>● Pull the perforated tab off of the box(es) of gloves.</li> <li>● From the top of the dispenser, slide glove box down into the dispenser with the opening of the box facing forward.</li> <li>● For the triple box dispenser, slide a glove box down into each of the dividers with the opening of the box facing forward.</li> <li>● Pull gloves out of the box one at a time.</li> <li>● Dispose of after single use.</li> </ul>
<b>Interpretation of Test Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	<ul style="list-style-type: none"> <li>● Use a soft clean with warm water, and mild soap.</li> <li>● Durable enough to use disinfecting wipes.</li> </ul>

<b>Cleaning – Automated</b>	
<b>Disinfection</b>	<ul style="list-style-type: none"> <li>● Acceptable agents: <ol style="list-style-type: none"> <li>1. Chlorine bleach</li> <li>2. Isopropyl alcohol</li> <li>3. Hydrogen peroxide</li> <li>4. Quaternary ammonia wipes</li> </ol> </li> </ul>
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	
<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	ProSys™ Brochure, ProSys™ Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

2020-08-28 Suzanne Latta