

## Instructions for Use: Healthmark Watermark™

<b>Brand Name of Product</b>	Healthmark Watermark™
<b>Generic Name of Product</b>	Water level guide and temperature gauge
<b>Product Code Number(s)</b>	TEMP-WMK
<b>Purpose of Product</b>	To indicate the appropriate water volume and temperature in a liquid bath.
<b>Range of Applications for Product</b>	When submerging items in a liquid bath
<b>Key Specifications of product</b>	

Shipping & Storage	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

Instructions for Using Product	
<b>Description of Use (s)</b>	To gauge the appropriate fill line and temperature of the water.
<b>Preparation for Use</b>	<ol style="list-style-type: none"> <li>1. Be sure the surface for adherence is dry.</li> <li>2. Read instructions from detergent manufacture and the cleaning equipment or sink.</li> <li>3. Record the volume of water in the white box on the Healthmark Watermark</li> </ol>
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Determine the appropriate fill line in the tank where the items are to be submerged.</li> <li>2. Remove adhesive label on the back.</li> <li>3. Place watermark on water fill line.</li> <li>4. Add cleaning solution according the instructions of the detergent manufacture.</li> <li>5. Staff should then be instructed as to the amount of detergent to add to the bath, in accordance with the detergent manufacturer's IFU.</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	

Reprocessing Instructions	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	The Watermark should be cleaned with the same products used to clean the sink/basin.
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	The Watermark should be disinfected with the same products as used to disinfect the sink.
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	<ul style="list-style-type: none"> <li>• Replace the Healthmark Watermark™ when adhesive back wears off and Watermark begins to peel.</li> <li>• After long term use, the liquid crystals may lose their elasticity and the color change will lose its vibrancy. Change the Watermark should this happen.</li> </ul>
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	

<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	ProFormance™ Brochure, ProFormance™ Price List
<b>Reference Documents</b>	

<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com
---------------------------------	---