



healthmark
INDUSTRIES CO.
health care products

33671 Doreka
Fraser, Michigan 48026
Telephone: (586) 774-7600
Fax: (586) 774-6473
www.HMARK.com

INSTRUCTIONS FOR USE 605LNG 605LNG w/Dots

*An unique seal for
sterilisation containers*

SEAL[®]Label

*An universal seal with
a label holder.*

*The label holder folds & clips
on the seal and is positioned on
the front side of any container.*



*The oval window allows you to check
if the SEAL[®]Label is correctly locked.*



*The electronic traceability label
is stuck on the SEAL[®]Label holder.*



Apply & Remove

1. Place arrow through slot of item to be sealed.
2. Gently push arrow into locking barrel until it engages locking teeth.
3. Gently push on padlock seal to ensure it has been properly locked.
4. To remove, hook forefinger under locking barrel and pull laterally away from the slot. **DO NOT PULL DOWNWARD**