



Instructions for Use: Personal Device Protection Covers

Brand Name of Product	Personal Device Protection Covers
Generic Name of Product	Disposable Plastic Protector
Product Code Number(s)	2005758, SES-002, DPPC-001, DPTC-002
Purpose of Product	Designed to help protect from moisture, dust and other contaminants.
Range of Applications for Product	For use on phone, tablet or keyboard.
Key Specifications of Product	

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product	
Description of Use(s)	Use disposable, zip top plastic sleeves to keep phones and tablets clean. Use keyboard cover to help protect from contaminants.
Preparation for Use	
Diagrams (drawings, pictures)	 
Steps for Use of Product	<p>Phone</p> <ol style="list-style-type: none"> 1. Slide phone or tablet inside cover. 2. Once inside, zip the bag closed and make sure it is secure. <p>Keyboard</p> <ol style="list-style-type: none"> 1. Take the cover out of package. 2. Cover the entire keyboard with protection cover.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	Damaged covers should be replaced immediately.
Disposal	Dispose as needed in accordance with your facility's hazardous materials disposal policy.

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	Cleaning should be performed when cover is completely pulled over the keyboard. Take a slightly moist cloth using warm water and facility approved cleaning solution, or 70% Isopropyl Alcohol, and wipe cover thoroughly. Do not submerge in water or wipe cover.
Cleaning – Automated	

Disinfection	
Drying	
Maintenance, Inspection and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	SST Brochure, SST Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2020-07-01 Matt Smith