

Instructions for Use: Pouch Sterilization Rack

Brand Name of Product	Pouch Sterilization Rack
Generic Name of Product	Sterilization Rack
Product Code Number(s)	SR-048, SR-010, SR-040, SR-004, SR-025, SR-005, SR-008, SR-016, SR-020, SPR-201206
Purpose of Product	Keeps pouches on edge. Prevents over stacking, facilitates sterilant contact, and reduces drying time (all in accordance with <i>AAMI ST79</i>).
Range of Applications for Product	N/A
Key Specifications of Product	<ul style="list-style-type: none"> Constructed of stainless-steel rod and tungsten (inert gas welded) for extra strength. Different sizes and configurations offered for customizable requirements. Styles include: <ul style="list-style-type: none"> Full loop Peg Multi-height racks.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Peel pouch racks are designed for optimal sterilant (for the sterilization cycle) and air flow (for complete drying of peel pouches).
Preparation for Use	N/A
Diagrams (drawings, pictures):	N/A
Steps for Use of Product	<ol style="list-style-type: none"> Place sterilization rack in the designated area of the sterilizer. Place each sealed pouch in each slot so that there is space between each pouch. When sterilization is complete, remove each pouch from the rack.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	<ul style="list-style-type: none"> Wash with a mild detergent. Rinse thoroughly.
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	<ul style="list-style-type: none"> Compatible with steam sterilization: <ul style="list-style-type: none"> 132 °C (270 °F) for 4 minutes 135 °C (275 °F) for 3 minutes. Materially compatible with EO and gas plasma/VHP sterilization.
Storage	N/A

Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	Sterilization Brochure, Sterilization Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com