







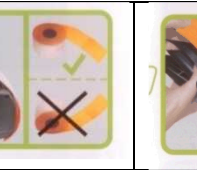



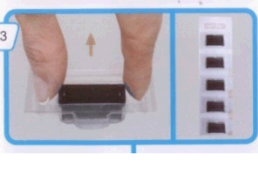




## Instructions for Use: Handheld Labeling System

<b>Brand Name of Product</b>	Handheld Labeling System
<b>Generic Name of Product</b>	Label Gun and labels
<b>Product Code Number(s)</b>	15.400.250, 15.500.515, 15.500.516, 15.500.517, 15.500.518, 15.500.519, 15.500.523, 15.500.524, 15.500.525
<b>Purpose of Product</b>	Handheld-Labeling System has been designed specifically with Central Sterile in mind for the labeling of any items that you need to be autoclaved - from sterile linen packs to instrument trays, and peel pouches.
<b>Range of Applications for Product</b>	Great versatility for labeling in other hospital departments, such as dietary and pharmacy.
<b>Key Specifications of Product</b>	Prints two-part labels designed to allow the sterilization record label to go with the patient record after use.

Shipping & Storage	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	Eighteen to twenty-four (18–24) months from date of manufacture.

Instructions for Using Product	
<b>Description of Use(s)</b>	Label Gun is used for labeling a variety of products and packages being autoclaved.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	<p><b>Setting the characters</b></p>   <p>1 2</p> <p><b>Labeling</b></p>     <p>1 2 3 4</p> <p><b>Replacing the label roll</b></p>    <p>1 2 3</p>  <p>4</p>

	<p><b>Replacing the ink-roll</b></p>     
<p><b>Steps for Use of Product</b></p>	<p><b>Setting the characters</b></p> <ol style="list-style-type: none"> <li>1. Pull adjustment knob to desired position and turn knob to select character to print.</li> <li>2. Press knob in before printing. (<b>Important:</b> Failure to do so before operating could cause damage.)</li> </ol> <p><b>Labeling</b></p> <ol style="list-style-type: none"> <li>1. Place your hand around the handle and squeeze your fingers towards your palm.</li> <li>2. Release the handle back to the starting position.</li> <li>3. Pulling the handle will release the label from the opposite end of the label gun.</li> <li>4. Repeat to use another label on another product or package.</li> </ol> <p><b>Replacing the label roll</b></p> <ol style="list-style-type: none"> <li>1. Pull red release for the top slide back.</li> <li>2. Place new label roll inside with the label flap coming over the top.</li> <li>3. Secure the end of the label roll to the top end of the label gun.</li> <li>4. When in use, the label will dispense through the chamber.</li> </ol> <p><b>Replacing the ink roll</b></p> <ol style="list-style-type: none"> <li>1. Pull down the back flap of the label gun.</li> <li>2. Raise the ink roll and pull outward. (The side notches keep the ink roll in place. Pushing the back end of the ink roll downward will also release it. We recommend wearing gloves. Your hands will get ink on them.)</li> <li>3. Grab a new ink roll.</li> <li>4. Place the new ink roll into the label gun as pictured. (Sided notch will click in place when secured).</li> <li>5. Close the back flap of the label gun.</li> </ol>
<p><b>Interpretation of Results</b></p>	<p>N/A</p>
<p><b>Contraindications of Test Results</b></p>	<p>N/A</p>
<p><b>Documentation</b></p>	<p>N/A</p>
<p><b>Special Warnings and Cautions</b></p>	<ul style="list-style-type: none"> <li>• <b>DO NOT PLACE THE LABEL GUN IN ANY STERILIZER.</b></li> <li>• To avoid damaging the Label Gun, press the adjustment knob after character selection and before printing.</li> </ul>
<p><b>Disposal</b></p>	<p>N/A</p>

Reprocessing Instructions	
<p><b>Point of Use</b></p>	<p>N/A</p>
<p><b>Preparation for Decontamination</b></p>	<p>N/A</p>
<p><b>Disassembly Instructions</b></p>	<p>N/A</p>
<p><b>Cleaning – Manual</b></p>	<p>Use a damp cloth to wipe down the Label Gun.</p>
<p><b>Cleaning – Automated</b></p>	<p>N/A</p>
<p><b>Disinfection</b></p>	<p>N/A</p>
<p><b>Drying</b></p>	<p>N/A</p>

<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	
<b>Sterilization</b>	All of the labels are compatible with Steam and EO sterilization. <b>DO NOT PLACE THE LABEL GUN IN ANY STERILIZER.</b>
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A

<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	Labeling Brochure, Labeling Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com