

Brand Name of Product	Preprinted Labels
Generic Name of Product	Labels
Product Code Number(s)	CSW-116, CSW-1216, CSW-254, CSW-289, CSW-302. CSW-IMP, CSW-RCS, CSW-SPU, CSW-SSS, DC-100214-71-BL, DC-EXPIRES, BIO-1X1, BIO-3X3
Purpose of Product	To communicate a variety of messaging on a label.
Range of Applications for Product	Place on equipment, trays, or anywhere key communication is required.
Key Specifications of Product	Adhesive backing options: <ul style="list-style-type: none"> • Permanent • Removable • Ultra-Removable • VH2O2 (hydrogen peroxide vapor [HPV]) Compatible.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	Store in room temperature.
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Peel off adhesive back and place Preprinted Label(s) firmly on equipment, trays, or desired location.
Preparation for Use	<ul style="list-style-type: none"> • Clean and dry the surface which the Preprinted Label will be applied to. • The surface should be at room temperature. • Peel off the adhesive back from the Preprinted Label to apply it to the surface.
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> 1. Peel adhesive from the back of the Preprinted Label. 2. Place the label on the desired location. 3. Use two (2) fingers to press firmly from the middle and move your fingers outward to the edges of the label.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • Do NOT use Preprinted Labels on items that are going to be sterilized, unless it is VH2O2 compatible adhesive (NOTE: In this case, check the item manufacturer's IFU for reprocessing instructions). • Unless permanent, replace the Preprinted Labels monthly.
Disposal	Dispose of used Preprinted Labels according to your facility's guidelines.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A

Related Healthmark Products	N/A
Other Product Support Documents	N/A
Reference Documents	Label Brochure
Customer Service Contact	Healthmark, A Getinge company 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com