



**Instructions for Use: Quick Turnover 2-Part Clean-Dirty Label**

<b>Brand Name of Product</b>	Quick Turnover 2-Part Clean-Dirty Label
<b>Generic Name of Product</b>	Tray tag
<b>Product Code Number(s)</b>	HM-52489
<b>Intended Use</b>	To identify the status of items during the transportation cycle. Label contains pull tags that indicate if items are clean or dirty and/or items needed for quick turnover.
<b>Range of Applications for Product</b>	Case carts or anything used to transport items during reprocessing.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>● 3.125" x 5.125" Perforated Tags</li> <li>● Adhesive Backing</li> <li>● Two Tags: <ul style="list-style-type: none"> <li>- Top Tag <ul style="list-style-type: none"> <li>● Transportation Tag, Quick Turnover Tag</li> </ul> </li> <li>- Bottom Tag <ul style="list-style-type: none"> <li>● Clean Tag, Dirty Biohazard Tag</li> </ul> </li> </ul> </li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	Keep in dry, room temperature area.
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	The Quick Turnover Label can be used on any piece of equipment/cart that is assigned to transport supplies with the potential to return containing contaminated items and/or for quick turnover.
<b>Preparation for Use</b>	Should be applied to a clean, dry surface.
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Removing label from the plastic backing: <ul style="list-style-type: none"> <li>● Before placing the label on the transportation cart/bin, slightly bend the bottom ½" of the label on the perforation. This will assist in the removal of the label in decontamination by creating a free piece of the label to grab.</li> </ul> </li> <li>2. Placing the label on the cart/bin: <ul style="list-style-type: none"> <li>● The label should be placed in a visible spot to facilitate observation by staff. This is typically on the door or top of the transportation cart/bin.</li> </ul> </li> <li>3. After the procedure: <ul style="list-style-type: none"> <li>● Hands should be covered with a clean pair of gloves for removal of the sticker.</li> <li>● Remove the TRANSPORTATION TAG by pulling up where it says 'Pull Here' in a right to left fashion to reveal the QUICK TURNOVER label underneath.</li> <li>● Write what items are needed for quick turnover and the time needed before returning to decontamination.</li> <li>● Remove the CLEAN tab by pulling up where it says 'Pull Here' in a right to left fashion to reveal the DIRTY BIOHAZARD label underneath.</li> </ul> </li> <li>4. Removing the label in decontamination: <ul style="list-style-type: none"> <li>● Prior to removing the label, read and identify what items are needed for quick turnover.</li> <li>● Remove the label by grabbing the bottom area just below the perforation. Pull in a slow motion up and away from the cart. DO NOT PULL FAST. Residue may be left behind if pulled too quickly.</li> <li>● Label can be discarded in the garbage.</li> </ul> </li> </ol>

<b>Interpretation of Test Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	The Quick Turnover Label is a single-use label and is to be discarded after use in a facility approved waste container according to facility policy.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	
<b>Related Healthmark Products</b>	Transportation Identification Label
<b>Other Product Support Documents</b>	Labeling Brochure, Labeling Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>