


Instructions for Use: SHL Face Shield with Drape

Brand Name of Product	Face Shield with Drape
Generic Name of Product	Face Shield with Drape
Product Code Number(s)	SHL-002
Intended Use	Designed to help protect the wearer from splash.
Range of Applications for Product	Areas where protection against splash is indicated.
Key Specifications of Product	<ul style="list-style-type: none"> • Latex-free elastic headband • Antistatic face shield • Face shield dimensions: 13 in. (H) x 9 in. (W) • 7 mil thick clear polyester (PET) • One inch (25.4 mm) thick-polyurethane brow foam • Drape dimensions- 12 inches in Length (300 mm) x 12 inches in Width (300 mm) • Independently tested and meets the minimum liquid barrier performance criteria as specified in ASTM F2100.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	Product must be stored out of sunlight and in a clean, dry (between 30% rh and 60% rh area maintained at a temperature between 10 °C and 30 °C (50 °F and 86 °F).
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Designed to help protect the wearer from splash.
Preparation for Use	N/A
Diagrams (drawings, pictures)	 <p>Areas of Shield in Direct Contact with Wearer:</p> <ul style="list-style-type: none"> • Forehead • Above Ears • Around Back of Head • Chest, when drape is tucked
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place foam front against your forehead. 2. Secure elastic band around the back of your head. 3. Tuck the drape into torso garment so that it covers the front facing neck area.
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • This face shield or any PPE is to be used according to your specific facility policy.

	<ul style="list-style-type: none"> • The face shield alone will not prevent infection from microbes or viruses nor is it effective against radiation. • Healthcare facilities are responsible for decisions regarding appropriate PPE for their personnel.
Disposal	Single-use

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	Product must be stored out of sunlight and in a clean, dry (between 30% rh and 60% rh) area maintained at a temperature between 10 °C and 30 °C (50 °F and 86 °F).
Additional Information	Independent test results are available upon request.
Related Healthmark Products	N/A
Other Product Support Documents	SST Brochure, SST Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-12-01 Suzanne Latta