


Brand Name of Product	Face Shield with Drape
Generic Name of Product	Face Shield with Drape
Product Code Number(s)	SHL-002
Intended Use	Designed to help protect the wearer from splash.
Range of Applications for Product	Areas where protection against splash is indicated.
Key Specifications of Product	<ul style="list-style-type: none"> • Latex-free elastic headband • Anti-static face shield • Anti-fog • Face shield dimensions: 13- (H) x 9 inches (W). • Seven (7) mil thick clear polyester (PET). • One (1)-inch (25.4 mm) thick-polyurethane brow foam. • Drape dimensions: 12- (L) x 12 inches (W) (300- [L] x 300 mm [W]). • Independently tested and meets the minimum liquid barrier performance criteria as specified in <i>ASTM F2100</i>.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	Product must be stored out of sunlight and in a clean, dry (between 30% rh and 60% rh area maintained at a temperature between 10- to 30 °C (50- to 86 °F).
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Designed to help protect the wearer from splash.
Preparation for Use	N/A
Diagrams (drawings, pictures)	 <p>Areas of Shield in Direct Contact with Wearer:</p> <ul style="list-style-type: none"> • Forehead • Above Ears • Around Back of Head • Chest, when drape is tucked
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place foam front against your forehead. 2. Secure the elastic band around the back of your head. 3. Tuck the drape into your torso garment so that it covers the front facing neck area.
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A

Special Warnings and Cautions	<ul style="list-style-type: none"> • This face shield (or any Personal Protective Equipment [PPE]) is to be used according to your specific facility policy. • The face shield alone will not prevent infection from microbes or viruses nor is it effective against radiation. • Healthcare facilities are responsible for decisions regarding appropriate PPE for their personnel.
Disposal	Single use.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	Product should be stored out of sunlight and in a clean, dry area. Long term recommended storage temperature is between 10- and 30 °C (50- and 86 °F).
Additional Information	Independent test results are available upon request.
Related Healthmark Products	N/A
Other Product Support Documents	SST Brochure, SST Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com