


Instructions for Use: SHL Face Shield

Brand Name of Product	Face Shield
Generic Name of Product	Face Shield
Product Code Number(s)	SHL-001, SHL-001-10
Intended Use	Designed to help protect the wearer from splash.
Range of Applications for Product	Areas where protection against splash is indicated.
Key Specifications of Product	<ul style="list-style-type: none"> • Latex-free elastic headband • Anti-static face shield • Anti-fog • Face shield dimensions: 13-(H) x 9 inches (W). • Seven (7) mil thick clear polyester (PET). One (1)-inch (25.4 mm) thick-polyurethane brow foam • Independently tested and meets the liquid barrier criteria, as specified in <i>ASTM F2100</i>.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	Product must be stored out of sunlight and in a clean, dry (between 30% – 60% rh) area that is maintained at a temperature between 10- and 30 °C (50- and 86 °F).
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Designed to help protect the wearer from splash.
Preparation for Use	N/A
Diagrams (drawings, pictures)	 <p>Areas of Shield in Direct Contact with Wearer:</p> <ul style="list-style-type: none"> • Forehead • Above Ears • Around Back of Head
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place foam front against your forehead. 2. Secure the elastic band around the back of your head.
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • This face shield (or any Personal Protective Equipment [PPE]) is to be used according to your specific facility's policy.

	<ul style="list-style-type: none"> • The face shield alone will not prevent infection from microbes or viruses nor is it effective against radiation. • Healthcare facilities are responsible for decisions regarding appropriate PPE for their personnel.
Disposal	Single use.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	Product should be stored out of sunlight and in a clean, dry area. Long term recommended storage temperature is between 10- and 30 °C (50- and 86 °F).
Additional Information	Independent test results are available upon request.
Related Healthmark Products	N/A
Other Product Support Documents	SST Brochure, SST Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com