



Instructions for Use: STA-DRI Disposable Jumpsuit

Brand Name of Product	STA-DRI Disposable Jumpsuit
Generic Name of Product	Disposable Jumpsuit
Product Code Number(s)	STA-DRI
Intended Use	To help keep technician dry when reprocessing dirty items in the decontamination area.
Range of Applications for Product	To protect and keep you dry from organic soil and water splash up.
Key Specifications of Product	<ul style="list-style-type: none"> • Non-skid disposable jumpsuit. • One size fits all. • Made of impermeable polyethylene for use with gown.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	To keep technicians protected from splashing and protect against organic soils.
Preparation for Use	Remove jumpsuit from packaging prior to use.
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<p>Donning:</p> <ol style="list-style-type: none"> 1. Unfold jumpsuit. 2. Place one foot into the leg down into the boot section of the jumpsuit. 3. Then place the other foot into the second leg of the jumpsuit into the boot section. 4. Make sure your feet are securely placed in the boot of the jumpsuit. 5. Pull the jumpsuit up the legs and body and pull the neck strap over the head to rest on the back of the neck. <p>Doffing:</p> <ol style="list-style-type: none"> 1. With gloves still on, pull neck strap back over head. 2. Pull jumpsuit down to feet and remove feet one at a time from boot cover. 3. Properly dispose of jumpsuit.
	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	Single-use, treat as biohazard. Dispose of in accordance with facility policy for contaminated items.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A

Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	PPE Brochure, PPE Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2021-06-14 Suzanne Latta