



**Instructions for Use: Scope Sleeve**

<b>Brand Name of Product</b>	Scope Sleeve
<b>Generic Name of Product</b>	Scope Sleeve
<b>Product Code Number(s)</b>	SSLV-003, SSLV-004, SSLV-005
<b>Intended Use</b>	Designed to cover and protect the insertion tube of the flexible endoscopes during storage and transportation and are made of absorbent material that also provides a barrier to environmental contaminants.
<b>Range of Applications for Product</b>	Protecting the insertion tube of the flexible endoscope.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>● Openings at both ends of the scope sleeve.</li> <li>● Absorbent and repellant to water.</li> <li>● Scope Sleeve dimensions-             <ul style="list-style-type: none"> <li>● SSLV-003- 85L x 5W centimeters</li> <li>● SSLV-004- 115L x 5W centimeters</li> <li>● SSLV-005- 190L x 5W centimeters</li> </ul> </li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	Package of 100
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	Helping to protect insertion tube of scope from environmental contaminants.
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. After processing the scope, insert the distal end of the insertion tube into the Scope Sleeve and pull upwards to the working channel.</li> <li>2. Once the sleeve is on the scope, place the scope in a facility approved hanging cabinet until ready to use.</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	The Scope Sleeve should be of appropriate length, so it does not touch the floor of the storage cabinet, otherwise it is considered contaminated.
<b>Disposal</b>	Single use only. After removing the sleeve from the scope, dispose of in compliance with facility policy.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	

<b>Storage</b>	
<b>Additional Information</b>	Single-use item
<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	Sterilization Products
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>