



**Instructions for Use: Tamper Evident Probe Cover**

<b>Brand Name of Product</b>	Tamper Evident Probe Cover
<b>Generic Name of Product</b>	Probe Cover
<b>Product Code Number(s)</b>	PRCVR-001
<b>Intended Use</b>	Secure clean items in the bag for appropriate storage of items to avoid environmental contamination.
<b>Range of Applications for Product</b>	To help protect items from contamination.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Self-Seal</li> <li>• Tamper Evident</li> <li>• Low-density polyethylene</li> <li>• Printed with a horizontal perforation with tape strip on the lip of bag</li> <li>• 14 x 4.5 Inch</li> <li>• 3 Mil</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	Secure clean items in the bag for appropriate storage of items to avoid environmental contamination. Not a sterile barrier.
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures)</b>	<p style="text-align: center;"> <span>Figure 1</span> <span style="margin-left: 200px;">Figure 2</span> </p>

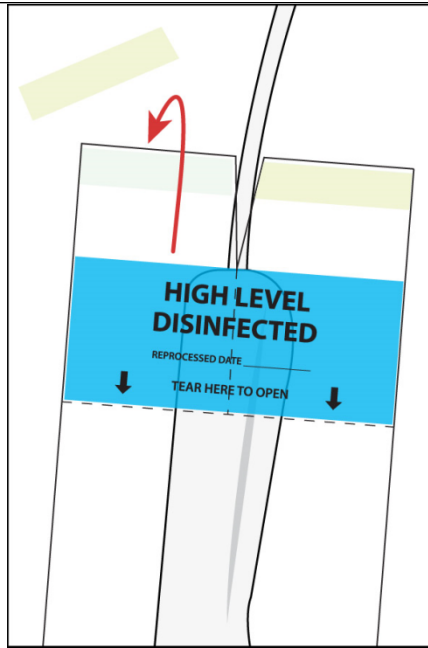


Figure 3

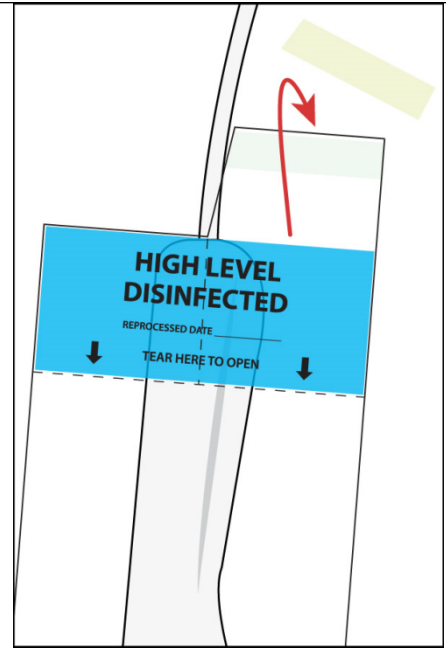


Figure 4

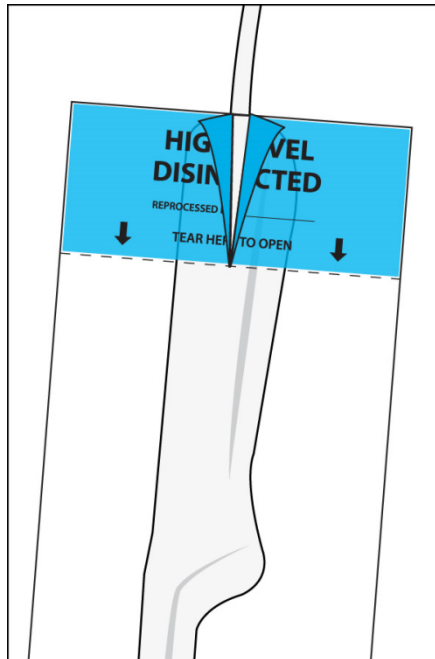


Figure 5

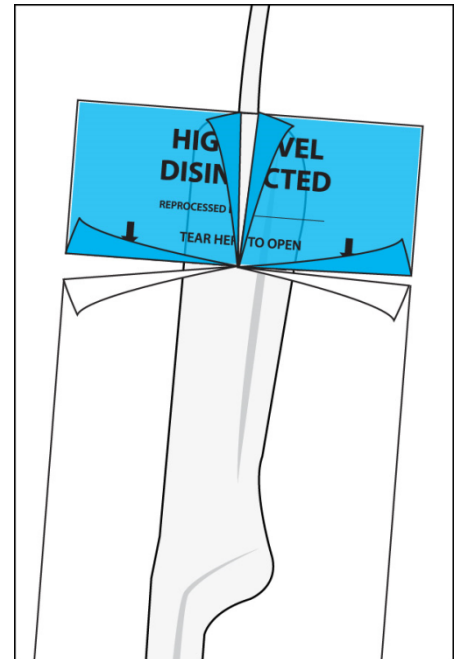


Figure 6

**Steps for Use of Product**

1. After disinfection, make sure the items are dry before placement in the Tamper Evident Probe Cover. If sterilization is used, items should be cool before placement in the probe cover.
2. Separate the vertical perforation halfway down to the top of the High-Level Disinfected label on the front side of the Tamper Evident Probe Cover. **Fig. 1**
3. Place the probe distal end first into the bag with the cord remaining outside of the Tamper Evident Probe Cover. **Fig. 2**
4. Remove the strip one at a time and fold the lip over to secure the seal. **Fig. 3, Fig. 4**
5. To open the probe cover, separate the vertical perforations. **Fig. 5**
6. Then pull across horizontally on each side of the probe cover. **Fig. 6**
7. Remove the probe.

**Interpretation of Test Results**

**Contraindications of Test Results**

**Documentation**

<b>Special Warnings and Cautions</b>	For items requiring a sterile barrier (see manufacturer's IFU), package in a sterile barrier system as directed.
<b>Disposal</b>	Single-use only

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	
<b>Related Healthmark Products</b>	Dust Covers
<b>Other Product Support Documents</b>	Sterilization Brochure, Sterilization Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>