



<b>Brand Name of Product</b>	TEMP-HL
<b>Generic Name of Product</b>	Temperature Monitoring
<b>Product Code Number(s)</b>	TEMP-HL
<b>Purpose of Product</b>	Irreversible, disposable thermometer has the ability to indicate both high and low temperatures to ensure that reprocessed items stay within a desired temperature range during transportation of goods.
<b>Range of Applications for Product</b>	Instrument sets and other equipment that has been reprocessed and requires temperature monitoring.
<b>Key Specifications of Product</b>	Irreversible Thermometer

**Shipping & Storage**

<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

**Instructions for Using Product**

<b>Description of Use (s)</b>	Thermometer indicates both high and low temperatures to ensure that reprocessed items stay within a desired temperature range during transportation of goods.
<b>Preparation for Use</b>	Be sure that instrument sets and equipment are reprocessed and dry before applying.
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Remove adhesive and place on the desired location where temperature is being monitored.</li> <li>2. When the temperature gets to 0°C, a distinct clear to red color change occurs. Conversely, when the temperature exceeds 29°C, the color changes from white to black.</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	<ul style="list-style-type: none"> <li>• If adhesive residue remains on the surface, may be removed with acetone or isopropyl alcohol solvent.</li> <li>• May be disposed of in regular trash.</li> </ul>

**Reprocessing Instructions**

<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	

<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	ProFormance™ Brochure, ProFormance™ Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd.

	Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com
--	---