



**Instructions for Use: Transport Trolley**

<b>Brand Name of Product</b>	Transport Trolley
<b>Generic Name of Product</b>	Transport Trolley
<b>Product Code Number(s)</b>	2220
<b>Intended Use</b>	To retrieve round soaker trays and transport them for reprocessing.
<b>Range of Applications for Product</b>	
<b>Key Specifications of Product</b>	

Shipping & Storage	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

**Instructions for Using Product**

**Description of Use (s)**  
**Preparation for Use**

**Parts Included For Assembly**


C: Pushhandle (1x)  
 E: Connecting tube (1x)  
 P: Connecting tube (1x)  
 G: Connecting tube (1x)  
 Q: Connecting tube (1x)  
 R: Top-shelf (1x)

B: Endo leftpart (1x)  
 A: Endo rightpart (1x)

H: Screw M5x12 (2x)  
 I: Screw M10x25 (8x)  
 J: Washer M5 (2x)  
 K: Covercap (8x)  
 L: Hexagonal wrench 3 mm (1x)  
 M: Hexagonal wrench 10 mm (1x)

**Diagrams (drawings, pictures)**

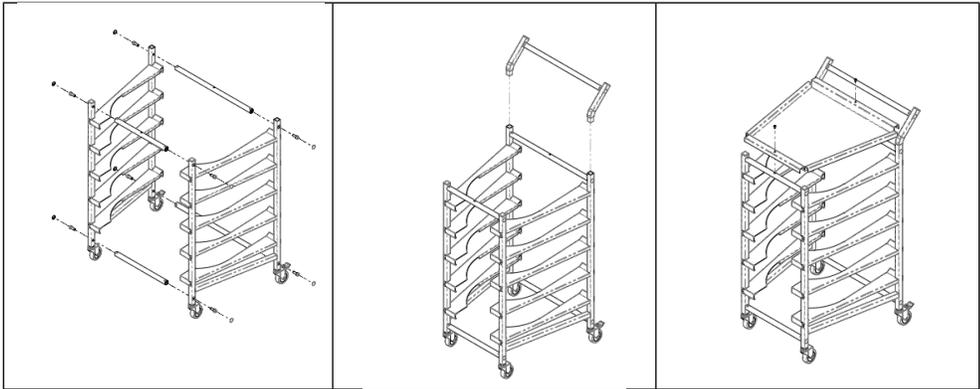

	 <div style="display: flex; justify-content: space-around; margin-top: 5px;"> <span>1</span> <span>2</span> <span>3</span> </div>
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Screw bolt I according drawing, using tool wrench M and place the cover cap K.</li> <li>2. Assemble frame as shown, using a plastic hammer to prevent frame from getting damaged. (part C)</li> <li>3. Assemble top shelf R as shown, using wrench L to screw the bolts H and washer J onto the trolley</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	Apply cleaning detergent with a moist towel.
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	

<b>Related Healthmark Products</b>	PPE Brochure, PPE Price List
<b>Other Product Support Documents</b>	
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>