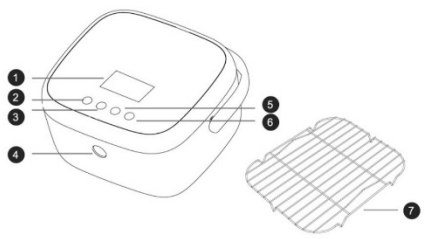



<b>Brand Name of Product</b>	UVC LED Boxes
<b>Generic Name of Product</b>	UVC LED Boxes
<b>Product Code Number(s)</b>	UVC-001, UVC-002
<b>Intended Use</b>	Sanitize personal items with UV-C light.
<b>Range of Applications for Product</b>	
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• 3-minute cycle</li> <li>• Wavelength between 260-280 nm</li> <li>• Portable</li> <li>• 2 different models- UVC-001- Dimensions- 11.8 x 10.5 x 6 inches Power adapter Power- 18V= 27W/1.5A Input Interface- Type-C USB Ozone free Smart drying Stainless steel rack</li> <li>UVC-002- Outer dimensions - 8.5 x 4 x 2.8 inches Inner dimensions- 8.5 x 3.5 x 1.5 inches Power adapter Power- 8W Reflective glass provides 360° for universal support CE, ROHS, and FCC certified</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	<ul style="list-style-type: none"> <li>• Lifespan of UV LED bulbs is 10,000 hours.</li> <li>• Product can be used for 3 years.</li> </ul>

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	Sanitize personal items with UV-C light.
<b>Preparation for Use</b>	Remove the UVC sanitizer from packaging.
<b>Diagrams (drawings, pictures)</b>	<p>UVC-001</p>  <ol style="list-style-type: none"> <li>1 Time Display Screen</li> <li>2 Fast Sterilize Button</li> <li>3 Sterilize&amp;Deodor+Smart Drying+Temp.control+Sterile Storage Button</li> <li>4 Pop up Button</li> <li>5 Sterilize&amp;Deodor+Sterile Storage Button</li> <li>6 ON/OFF Button</li> <li>7 Stainless steel tool holder</li> </ol>

	<p><b>UVC-002</b></p> <p style="text-align: center;"><b>Disinfection and Storage 2 in 1</b></p> 
<p><b>Steps for Use of Product</b></p>	<p><b>UVC-001</b></p> <ol style="list-style-type: none"> <li>1. Press the ON/OFF button on top of lid to ON.</li> <li>2. Open the lid and place items on the rack.</li> <li>3. Close the lid.</li> <li>4. Press the desired cycle.</li> <li>5. When finished, press the ON/OFF button to turn OFF.</li> </ol> <p><b>UVC-002</b></p> <ol style="list-style-type: none"> <li>1. Plug in the UVC box.</li> <li>2. Open the lid and place the item in the box and close the lid. You will hear a voice message.</li> <li>3. Press the one-button start function for 0.5 seconds.</li> <li>4. After 3 minutes, the UVC box will automatically turn off and a voice will prompt (completed) to let you know sanitation is complete.</li> <li>5. Do not open lid during sanitation.</li> </ol>
<p><b>Interpretation of Test Results</b></p>	
<p><b>Contraindications of Test Results</b></p>	
<p><b>Documentation</b></p>	
<p><b>Special Warnings and Cautions</b></p>	<p>Not for use with any healthcare product. Intended only for personal-use items (cell phone, keys, etc.).</p>
<p><b>Disposal</b></p>	

<p><b>Reprocessing Instructions</b></p>	
<p><b>Point of Use</b></p>	
<p><b>Preparation for Decontamination</b></p>	
<p><b>Disassembly Instructions</b></p>	
<p><b>Cleaning – Manual</b></p>	
<p><b>Cleaning – Automated</b></p>	
<p><b>Disinfection</b></p>	
<p><b>Drying</b></p>	
<p><b>Maintenance, Inspection, and Testing</b></p>	<ul style="list-style-type: none"> <li>• UVI-001 indicators from Healthmark can be used to verify exposure to UV-C light during the cycle.</li> <li>• Run two consecutive cycles (6 minutes) with the UVI-001 to compare to the color interpretation chart.</li> </ul>
<p><b>Reassembly Instructions</b></p>	
<p><b>Packaging</b></p>	
<p><b>Sterilization</b></p>	
<p><b>Storage</b></p>	
<p><b>Additional Information</b></p>	
<p><b>Related Healthmark Products</b></p>	

<b>Other Product Support Documents</b>	PPE Brochure, PPE Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>