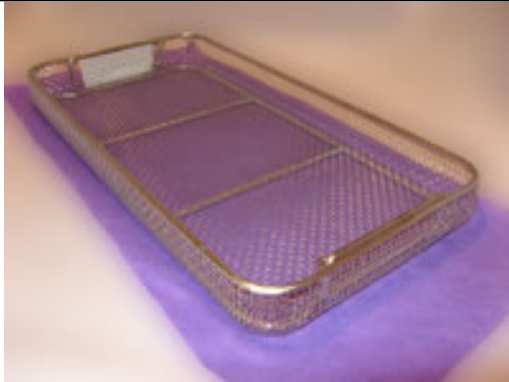


Brand Name of Product	UnderGuard for Plasma™
Generic Name of Product	Tray Liners
Product Code Number(s)	MT-2030, MT-1520, MT-1224
Purpose of Product	For use under the tray to protect outer wrap from sharp corners and imperfections of sterilization trays.
Range of Applications for Product	N/A
Key Specifications of Product	<ul style="list-style-type: none"> Manufactured from spun polypropylene. The unique purple color helps distinguish this protective layer from sterilization wrap.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	<ul style="list-style-type: none"> 15- to 30 °C 30- to 70% rh
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	To protect outer wrap from sharp corners and imperfections of sterilization trays.
Preparation for Use	<ul style="list-style-type: none"> Select a liner which will best fit the target basket. If using with a wrapped tray, the liner must be inside the wrap and may be used inside or under the basket. If using with a closed container, the liner must be used inside the container, either inside or below the basket.
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> Place a liner either inside or beneath basket. (In the event you are using a container, the liner should be placed in case first.) Place items in tray. Use proper sterility assurance indicators and follow hospital protocol. Wrap or close package. Place in VH2O2, VH2O2-O3, or steam sterilizer.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	MT-XXXX are number 5 for recycling.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A

Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	Material compatible with VH2O2/O3 and steam sterilization up to 137 °C.
Storage	<ul style="list-style-type: none"> • 15- to 30 °C • 30- to 70% rh
Additional Information	N/A

Related Healthmark Products	N/A
Other Product Support Documents	Sterilization Brochure, Sterilization Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com