



Instructions for Use: Clamps & Spreaders

Brand Name of Product	Clamps & Spreaders
Generic Name of Product	Clamps & Spreaders
Product Code Number(s)	20411, 20408, 20406, 20412, 20407, T0200
Purpose of Product	To keep the item in an open position, while being cleaned and sterilized
Range of Applications for Product	<ul style="list-style-type: none"> • Clamps ensure joints of jaws of the bone forceps remain open. • Spreaders are for items unable to be dismantled with ring handles. The jaws remain open during cleaning and sterilization processes.
Key Specifications of Product	<ul style="list-style-type: none"> • Stainless Steel • Silicone Covering

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Keeps an item in an open position while being cleaned and sterilized.
Preparation for Use	Select the appropriate size clamp before using.
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> 1. Open item to the appropriate width. 2. Insert ends of the clamps or spreaders through the handle's loops and rings. 3. Continue normal procedures according to the manufacturer's guidelines.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	AAMI, AORN, device manufacturers, and other industry experts strongly recommend items should be kept in an open position for maximum exposure.
Special Warnings and Cautions	N/A
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	Can be cleaned through the automatic washer with other items.
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	<ul style="list-style-type: none"> • Reprocessing steam sterilization: <ul style="list-style-type: none"> ○ 4 minutes at 132 °C (270 °F) ○ 3 minutes at 135 °C (275 °F). • Steam sterilization with instruments: <ul style="list-style-type: none"> ○ 4 minutes at 132 °C (270 °F) ○ 3 minutes at 135 °C (275 °F). • Compatible with EO sterilization.
Storage	N/A
Additional Information	N/A

Related Healthmark Products	N/A
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com