









Brand Name of Product	Clean Bags
Generic Name of Product	Clean Bags
Product Code Number(s)	LB0609CL, LB0814CL, CL536-3M
Intended Use	Secure clean items in the bag for appropriate storing to avoid environmental contamination. (Note: <i>Not a sterile barrier.</i>)
Range of Applications for Product	Items that have gone through the final step of processing.
Key Specifications of Product	<ul style="list-style-type: none"> • Code numbers LB0609CL, LB0814CL <ul style="list-style-type: none"> ○ Self-seal ○ Tamper-evident • Code number CL536-3M <ul style="list-style-type: none"> ○ “Clean” is displayed on the bag. ○ Not self-seal.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Secure clean items in the bag for appropriate storage to avoid environmental contamination. <i>Not a sterile barrier.</i>
Preparation for Use	N/A
Diagrams (drawings, pictures)	<p>Clean Bag LB0814CL</p> <div style="display: flex; flex-wrap: wrap; justify-content: space-around;"> <div style="text-align: center;">  <p>Figure 1</p> </div> <div style="text-align: center;">  <p>Figure 1a</p> </div> <div style="text-align: center;">  <p>Figure 2</p> </div> <div style="text-align: center;">  <p>Figure 3</p> </div> </div>

	<p>Clean Bag CL536-M</p>   <p style="text-align: center;">Figure 4 Figure 4a</p>
Steps for Use of Product	<p>Code numbers LB0609CL, LB0814CL</p> <ol style="list-style-type: none"> 1. Ensure disinfected items are dry before placement in the bag. If sterilization is used, items should be cool before placement in bag. 2. Peel the white strip away from the Clean Bag and fold the adhesive section of the Clean Bag over. (Fig.1, 1a). 3. Press your fingers together from the middle and slide outward to the edge to seal the bag. (Fig.2). It will take around three (3) minutes for the seal to set up. 4. Place in storage until use. 5. To open the Clean Bag, tear along the perforated line at the bottom of the bag to obtain the contents. (Fig.3). <p>Code number CL536-3M</p> <ol style="list-style-type: none"> 1. Ensure disinfected items are dry before placement in the bag. If sterilization is used, items should be cool before placement in bag. 2. Fold seal over bag and seal with tape or tamper-evident label. (Fig. 4, 4a).
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • The Clean Bag is <i>not a sterile barrier</i>. Items requiring a sterile barrier should be packaged according to its manufacturer's IFU. • Single-use only.
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A

Related Healthmark Products	Dust Covers
Other Product Support Documents	Sterilization Products Brochure, Sterilization Products Price List
Reference Documents	N/A
Customer Service Contact	<p>Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com</p>