

Brand Name of Product	Cool Aids Vest
Generic Name of Product	Vest
Product Code Number(s)	VEST-S, VEST-M, VEST-L, VEST-XL, VEST-XXL, VEST-XXXL, VEST-XXXXL CPX-001, VEST-S-WO-P, VEST-M-WO-P, VEST-L-WO-P, VEST-XL-WO-P, VEST- XXL-WO-P, VEST-XXXL-WO-P
Intended Use	To provide cooling of the core body temperature.
Range of Applications for Product	N/A
Key Specifications of Product	<ul style="list-style-type: none"> • V-neck w/zipper closure • Lightweight, easy to activate, and durable. • One set of four (4) cooling inserts is included with each vest and one (1) cooler bag. • Includes inside pockets for cooling inserts.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	To provide cooling of the core body temperature.
Preparation for Use	<ul style="list-style-type: none"> • Activate by placing phase change packs in a refrigerator or freezer to cool. • Simply place phase change packs in freezer for a minimum of 20 minutes, or place in the refrigerator for a minimum of 40 minutes. • Phase change packs freeze at 58 °F. • It is recommended to wear a thin undergarment, such as a cotton T-shirt, under the vest for comfort.
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ul style="list-style-type: none"> • Place each phase change pack (four [4] total) inside the mesh pockets inside the vest. • Wait 10 minutes before putting on the vest. • Place each arm into the vest, so it is secure around each shoulder. • Zip the vest completely up to your upper chest.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	<p><u>Vest</u></p> <ul style="list-style-type: none"> • Remove the phase change packs from the vest prior to manual cleaning. • Hand-wash in warm water using a mild soap. Rinse. Hang to dry. <p>NOTE: Cleaning/washing of these products should be done in a clean area (e.g., a locker room).</p> <p><u>Phase Change Packs</u> Clean with light soap and water.</p>
Cleaning – Automated	<p><u>Vest</u></p> <ul style="list-style-type: none"> • Remove phase change packs from vest prior to cleaning in a washing machine.

	<ul style="list-style-type: none"> • Use light soap and gentle cycle.
Disinfection	N/A
Drying	Item should then be hung up in a clean area to dry. Keep in a clean area between uses.
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A

Related Healthmark Products	N/A
Other Product Support Documents	PPE Brochure, PPE Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com