







## FIS-007U Packaging Instructions

### Removing the FIS-007U

		<ol style="list-style-type: none"> <li>1. To lift the outermost foam ring, put your thumb into the insert, lift and pull back (Do not remove the attached foam from the packaging).</li> <li>2. This is showing two connectors packaged in the case.</li> </ol>
		
		<ol style="list-style-type: none"> <li>3. Next, carefully lift the connector and the Flexible Inspection Scope™ and set aside.</li> </ol>

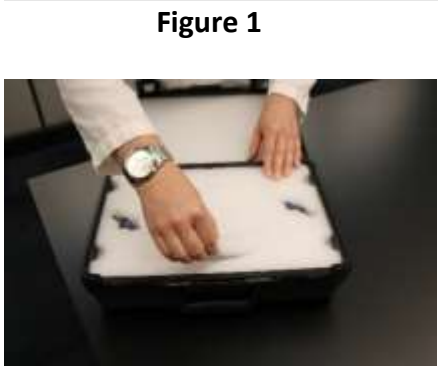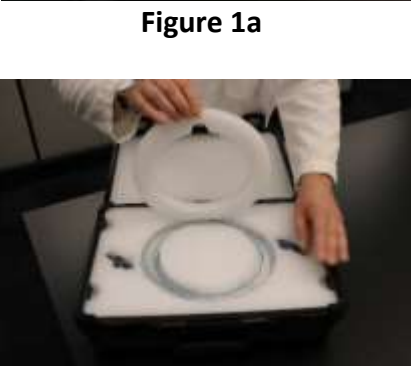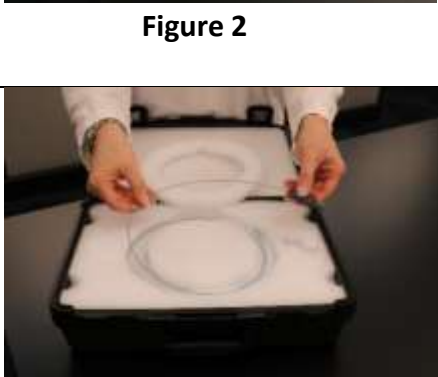

**Figure 4**



**Figure 4a**

4. If there are two Flexible Inspection Scopes™, lift the top connector and carefully remove the Flexible Inspection Scope™ and set aside. Lift the bottom connector and carefully remove the other Flexible Inspection Scope™ and set aside.



**Figure 5**



**Figure 5a**

5. Place the outer most foam ring back down. Lift the foam packaging and place on the lid of the case.



**Figure 5b**











**Figure 6**



**Figure 6a**

6. Lift the control box out of the foam using your index and middle fingers and set aside. Remove the cable cord, adhesive connectors, and thumb drive from the insert and set aside.

## Repackaging the FIS-007U

		<ol style="list-style-type: none"> <li>1. Place the cable cord, adhesive connectors, and thumb drive back into the foam insert. Using your index and middle fingers, place the control box back into the foam insert.</li> </ol>
		<ol style="list-style-type: none"> <li>2. Place the foam back down onto the cable cord and control box.</li> </ol>
		<ol style="list-style-type: none"> <li>3. Carefully place the Flexible Inspection Scope™ back in and around the inner most ring and the connector into the foam insert.</li> </ol>
		<ol style="list-style-type: none"> <li>4. If you have two connectors in the case, carefully place the Flexible Inspection Scope™ back in and around the inner most ring and the connector into the foam insert. Carefully place the second Flexible Inspection Scope™ on top of the other and place the connector into the other foam insert.</li> </ol>



**Figure 4b**



**Figure 4c**



**Figure 5**



**Figure 5a**



**Figure 5b**

5. Lastly, take the outer most foam ring and gently press down over the Flexible Inspection Scope™. Make sure it is secured.