



Instructions for Use: Foam Corner Protectors

Brand Name of Product	Foam Tray Corner Protectors
Generic Name of Product	Foam Corner Protectors
Product Code Number(s)	FP-1004
Intended Use	Foam corner protectors are designed to protect wrap from sharp corners and edges of trays, caused by shifting during handling and transport. Compatible with steam sterilization.
Range of Applications for Product	Instrument sets that require protection during transport and storage.
Key Specifications of Product	<ul style="list-style-type: none"> ● Open cell, reticulated foam ● Set of 4 corners

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	Foam corner protectors are designed to protect wrap from sharp corners and edges of trays.
Preparation for Use	Pull corners apart before placing on each corner of instrument tray.
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place one foam corner under each corner of tray. 2. Then place tray on top of foam protectors and make sure each corner of the tray sits on top of foam protectors.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	Single use only.
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for decontamination	
Disassembly Instructions	
Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	<ul style="list-style-type: none"> ● Steam sterilization at 270°F (132°C) for 4 minutes or 275°F (135°C) for 3 minutes. ● Maximum extended time for 30 minutes at 275°F (135°C). ● Compatible with EO sterilization.
Storage	
Additional Information	

Related Healthmark Products	
------------------------------------	--

Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	
Customer Service contact:	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com