



**Instructions for Use: Heat Mitts & Gloves**

<b>Brand Name of Product</b>	Insulated Gauntlet Mitts, Seamless Autoclave Gloves and Terry Cloth Autoclave Gloves
<b>Generic Name of Product</b>	Heat Mitts & Gloves
<b>Product Code Number(s)</b>	75574, 75524, 73224, JB-2636, JB-422-11
<b>Purpose of Product</b>	For removing hot trays and racks from autoclaves
<b>Range of Applications for Product</b>	Trays, racks and other hot items that are being removed from the autoclave
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>Insulated Gauntlet Mitts include protective insulation rated to 350°F.</li> <li>Seamless Autoclave Gloves are 100% cotton with exclusive high-density loop structure rated to 450°F.</li> <li>Terry Cloth Gloves are 100% cotton with gauntlet style cuffs rated to 250°F.</li> </ul>

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Insert hands completely into the mitts.</li> <li>2. Be sure that the mitts are completely covering your hands for maximum protection.</li> </ol>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	
<b>Disposal</b>	

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	<p>It is extremely important that you DO NOT use any bleach or dry-cleaning solution when cleaning these products.</p> <ol style="list-style-type: none"> <li>1. Rinse gloves/sleeves in cold water.</li> <li>2. If presoaking, use lukewarm water to ensure good particle softening. Mild detergents or chemicals should be used to enhance the laundering process.</li> <li>3. All products should be laundered using a mild soap and detergent.</li> <li>4. Use a warm water wash with a temperature not to exceed 160°F (71°C). Wash time should not be longer than 10 to 15 minutes.</li> <li>5. Rinse product in cold water.</li> <li>6. Repeat above as often as needed to ensure a thoroughly clean product. It is important to use a high extraction speed to thoroughly remove all excess moisture.</li> </ol>
<b>Disinfection</b>	
<b>Drying</b>	<ul style="list-style-type: none"> <li>Tumble dry all seamless knitted products in a temperature of not higher than 150°F (67°C) for a time of 10 to 15 minutes.</li> <li>Tumble dry terry products in the same conditions as above, but the time to dry may increase depending on the quantity and weight of the products. Temperatures should not exceed 150°F (67°C).</li> </ul>

<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	

<b>Related Healthmark Products</b>	Seamless Autoclave Gloves, Terry Cloth Autoclave Gloves, Shark Tooth Heat Mitts
<b>Other Product Support Documents</b>	ProSys™ Brochure, ProSys™ Price List
<b>Reference Documents</b>	
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>

msmith