


<b>Brand Name of Product</b>	LTA Disposable External Brush
<b>Generic Name of Product</b>	Disposable Brush
<b>Product Code Number(s)</b>	SU-510-1, SU-510-10, SU-510-50, SU-520-1, SU-520-10, SU-520-50, SU-530-1, SU-530-10, SU-530-50
<b>Intended Use</b>	Clean hard-surface items before processing.
<b>Range of Applications for Product</b>	Products that require precision cleaning of devices with hard external surfaces.
<b>Key Specifications of Product</b>	<p>Single-use brushes with latex-free thermoplastic handles and nylon bristles.</p> <ul style="list-style-type: none"> <li>• <i>SU-530</i>: <b>Soft</b>-fiber style brush is colored white with white bristles and is recommended for routine cleaning.</li> <li>• <i>SU-520</i>: <b>Medium</b>-fiber style brush is colored yellow with white bristles and is recommended for routine cleaning.</li> <li>• <i>SU-510</i>: <b>Rigid</b>-fiber style brush is colored green with white bristles and is recommended for intensive cleaning.</li> </ul>

Shipping & Storage	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	N/A

Instructions for Using Product	
<b>Description of Use(s)</b>	Designed for cleaning hard-surface items.
<b>Preparation for Use</b>	(Before each use), check the integrity and condition of the brush. Replace any worn or damaged brush.
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Consult and follow the manufacturer's IFU for the item you are cleaning, respectfully. (Note: Pay close attention to recommendations on the material composition of the brush trim.)</li> <li>2. Visually inspect instruments for {debris, residual, bioburden, blood, and/or protein}.</li> </ol>
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	N/A
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>• External brushes are intended as: <ul style="list-style-type: none"> <li>○ Cleaning tools only.</li> <li>○ Not in patient care.</li> <li>○ Not for aseptic techniques.</li> </ul> </li> <li>• Wear appropriate PPE in compliance with your facility's policy.</li> <li>• ☠ Treat used brushes as biohazardous (recommended).</li> </ul>
<b>Disposal</b>	It is recommended to dispose of used LTA Disposable External Brushes in a biohazard container since they are used to clean patient-used items.

**Reprocessing Instructions**

<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	N/A
<b>Storage</b>	N/A
<b>Additional Information</b>	Intended for single use. “Use” should be defined by facility policy.
<b>Related Healthmark Products</b>	Brush Cleaning Tools
<b>Other Product Support Documents</b>	ProSys™ Brochure, ProSys™ Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>