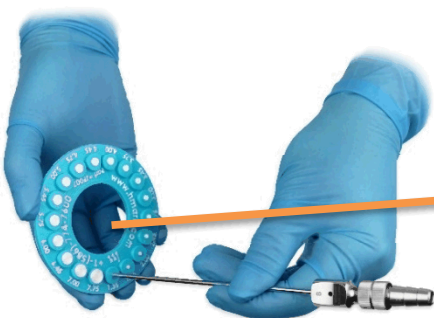



Lumen Sizer User Guide

Item # JP007

The Lumen Sizer helps to determine the internal diameter of a lumen opening to select the size brush for cleaning the lumen's internal surface.

1. Place the opening of the lumen over the Lumen Sizer's raised posts (**Fig. 1**) until the diameter size is determined.

	<p>Lumen Sizer</p> 	<p>Size in millimeters (mm)</p> <table border="1"> <tr> <td>1.35</td> <td>1.75</td> <td>2.00</td> </tr> <tr> <td>2.45</td> <td>2.75</td> <td>3.00</td> </tr> <tr> <td>3.45</td> <td>3.75</td> <td>4.00</td> </tr> <tr> <td>4.45</td> <td>4.75</td> <td>5.00</td> </tr> <tr> <td>5.45</td> <td>5.75</td> <td>6.00</td> </tr> <tr> <td>6.45</td> <td>7.00</td> <td>7.75</td> </tr> </table>	1.35	1.75	2.00	2.45	2.75	3.00	3.45	3.75	4.00	4.45	4.75	5.00	5.45	5.75	6.00	6.45	7.00	7.75
1.35	1.75	2.00																		
2.45	2.75	3.00																		
3.45	3.75	4.00																		
4.45	4.75	5.00																		
5.45	5.75	6.00																		
6.45	7.00	7.75																		
<p>Figure 1: (Lumen Sizer with raised white posts pictured above)</p>	<p>Figure 2: (Size Range)</p>																			

2. Select the corresponding sized brush based on the on the Lumen Sizer's range of sizes. (**Fig. 2**).
3. Insert the brush into the lumen to check for cleanliness of the internal surface.
 - a.

If debris is visible on the Lumen Sizer after sizing the lumen, send the Lumen Sizer back to decontam for reprocessing.

- *Manual Cleaning:* To manually clean, wipe down with disinfecting wipes.
- *Cleaning & Disinfection:* Use a neutral pH detergent. Place Lumen Sizer in a (secure) tray and place it on washer rack. Set the washer disinfecter on the "instrument cycle" and run.