

## Instructions for Use: Nic-Nac Washer Baskets & Balls

<b>Brand Name of Product</b>	Nic-Nac Washer Baskets & Balls
<b>Generic Name of Product</b>	Washer baskets for small items
<b>Product Code Number(s)</b>	NNB001, 21000-H, NNB002, NNB003, NIC000, NIC MICRO 1, NIC MICRO 2A, NIC MICRO 2, NIC MICRO 3, CIL-004, CIL-005.
<b>Purpose of Product</b>	To hold small items when being washed inside of an automated instrument washer, preventing damage and loss.
<b>Range of Applications for Product</b>	Small items that could be blown around, damaged or lost in an automated instrument washer.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Manufactured from stainless steel.</li> <li>• Hinged lid for safe containment of small parts.</li> <li>• Mesh design to allow thorough exposure to cleaning and rinsing solution within an automated instrument washer.</li> </ul>

Shipping & Storage	
<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	N/A
<b>Shelf Life</b>	N/A

Instructions for Using Product	
<b>Description of Use(s)</b>	Place in the washer basket small items that often get lost or damaged inside of an automated instrument washer.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	N/A
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Place small items in basket.</li> <li>2. Close lid of basket.</li> <li>3. Place into instrument washer for cleaning.</li> </ol>
<b>Interpretation of Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	N/A
<b>Special Warnings and Cautions</b>	N/A
<b>Disposal</b>	N/A

Reprocessing Instructions	
<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A
<b>Sterilization</b>	<ul style="list-style-type: none"> <li>• Compatible with steam sterilization.</li> <li>• Recommended cycle is 4 minutes at 132 °C (270 °F) or 3 minutes at 135 °C (275 °F) in a pre-vac sterilizer.</li> <li>• Maximum temperature is 135 °C (275 °F) for 30 minutes.</li> </ul>
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A

<b>Related Healthmark Products</b>	Washer Baskets
------------------------------------	----------------

<b>Other Product Support Documents</b>	ProSys™ Brochure, ProSys™ Price List
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> <a href="http://hmark.com">hmark.com</a>