




Instructions for Use: Non-Linting Disposable Wipes

Brand Name of Product	Non-linting Disposable Wipes
Generic Name of Product	Instrument Cleaning and Drying Wipes
Product Code Number(s)	NLW001, NLW002, NLW003
Purpose of Product	Non-linting wipes are for use in removal of gross soil immediately after the item has been used, during manual cleaning processes, and for drying prior to automated or manual disinfection.
Range of Applications for Product	Any item that requires: <ul style="list-style-type: none"> • Wiping as part of the point of use treatment • External wiping during the cleaning process. • Drying before further processing.
Key Specifications of Product	<ul style="list-style-type: none"> • Non-linting blended poly-cellulose material • Single-use

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Contents	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	To allow for safe cleaning and drying of sensitive items and fixtures as well as all general surfaces.
Preparation for Use	<ul style="list-style-type: none"> • Follow the item manufacturer's (Mfr.'s) instructions for use (IFU). • If utilizing a cleaning agent, be sure to follow the cleaning agent Mfr.'s IFU for proper preparation of the cleaning solution.
Diagrams (drawings, pictures)	
Steps for Use of Product	<p>When used for cleaning:</p> <ol style="list-style-type: none"> 1. Moisten with cleaning solution (if applicable). 2. Wipe down item. 3. Dispose after use on a single item. <p>When used for drying:</p> <ol style="list-style-type: none"> 1. Do not moisten. 2. Wipe down item to remove residual moisture. 3. Dispose after use on a single item.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • Consult the item Mfr.'s IFU to ensure compliance with their instructions. • Pay attention to area(s) of the item that <i>must</i> be free from moisture. • Pay attention to chemical compatibility of the item and areas of the item with the cleaning agent you select.
Disposal	When used in cleaning processes, treat as biohazard and discard of in compliance with facility guidelines.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A

Related Healthmark Products	DuraSponge™, Sterile Wipes
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com