



healthmark
INDUSTRIES CO.
health care products
800-521-6224
www.hmark.com


Instructions for Use: Pre-Rinse Shower Unit

Brand Name of Product	Pre-Rinse Shower Unit
Generic Name of Product	Shower Unit
Product Code Number(s)	300718, 300719, 301189, 301190, 313116, 313323
Purpose of Product	For safe and effective rinsing of gross contaminants.
Range of Applications for Product	
Key Specifications of Product	<ul style="list-style-type: none"> • Polished Chrome Plated Brass • Single Hole Deck Mount Pantry Faucet • 1/4 Turn Compression Valves w/ Check Valves • Lever Handles w/ SANIGUARD® Treatment • S/S Flex Hose w/ Santoprene Reinforced Hose • Stainless Steel Strain Relief – Patented • O-Ring Seals – Patented • Standard Full Spray Pattern • 1.2 GPM Spray Face included • Hold Open Ring and Hose Hook included • Commercial Specification Grade • Gaskets and Seals Rated Commercial Hot 180°F • 1/2” NPS Female Inlets • Operating Temperature: 40 - 180°F • Operating Pressure: 15 - 125psi

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	Rinsing of gross contaminates from surfaces.
Preparation for Use	Be sure to consult the manufacturer IFU of the item(s) to ensure that those items should be exposed to spray washing.

Diagrams (Drawings, Pictures)

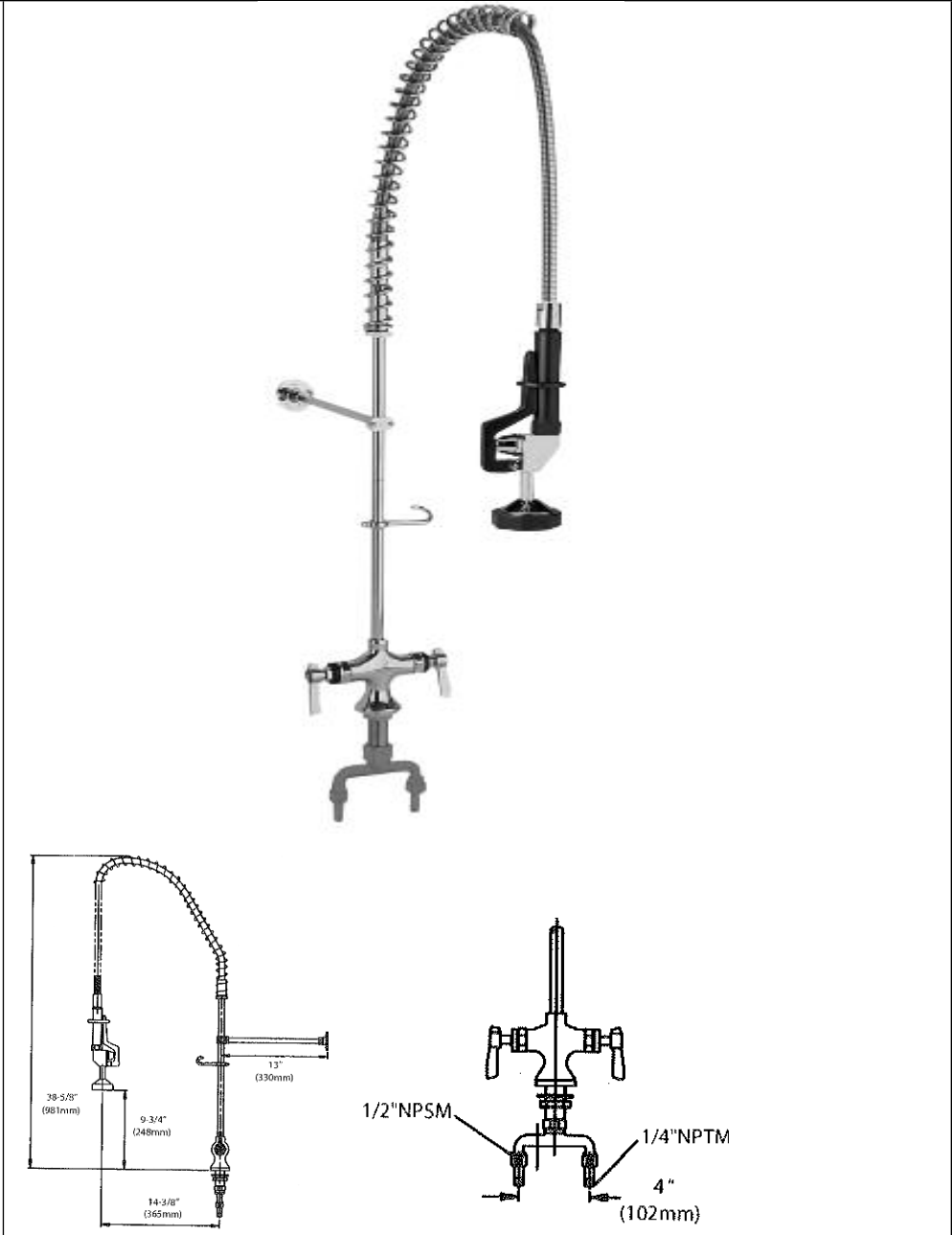

<p>Steps for Use of Product</p>	<ol style="list-style-type: none"> 1. Place item under spray unit. 2. Rinse thoroughly with spray to remove gross debris and soil.
<p>Interpretation of Results</p>	
<p>Contraindications of Test Results</p>	
<p>Documentation</p>	
<p>Special Warnings and Cautions</p>	<ul style="list-style-type: none"> • It is usually recommended to use water from the facility’s cold-water source for the initial rinse in order to avoid denaturing proteins and other organics. Consult the item manufacturer’s IFU. • The Shower Unit is not a medical product and therefore should not be used as part of medical treatments. • When in operation, the Shower Unit must not be pointed at parts or orifices of the body because the compressed water or air jet may be harmful. • Be sure to wear proper PPE according to facility and industry guidelines.
<p>Disposal</p>	

<p style="text-align: center;">Reprocessing Instructions</p>	
<p>Point of Use</p>	
<p>Preparation for Decontamination</p>	
<p>Disassembly Instructions</p>	

Cleaning – Manual	
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	
Storage	
Additional Information	Options include deck mount, splash mount, faucet add-on, wall bracket and an optional 36” long hose.

Related Healthmark Products	
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	Complies with: <ul style="list-style-type: none"> • ANSI A112.18.1M • CSA B125 • NSF 61 / 9 • ASTM F2324
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com