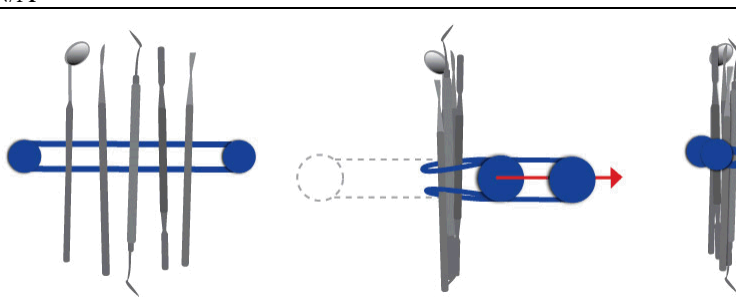


Instructions for Use: Pro-Ties™

Brand Name of Product	Pro-Ties™
Generic Name of Product	Bundling tool
Product Code Number(s)	310-054, 310-053, 310-052, 310-051
Purpose of Product	Pro-Ties™ are used to simplify the process of bundling items. They can significantly reduce time spent sorting.
Range of Applications for Product	
Key Specifications of Product	<ul style="list-style-type: none"> • Stretches to three times its original size • They will not stretch out of shape, rust, or breed bacteria • Steam sterilizable • Reusable • 4 colors: <ul style="list-style-type: none"> ○ 310-054 Yellow ○ 310-053 Blue ○ 310-052 Green ○ 310-051 Red

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	6 ties per box
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Pro-Ties™ are used to simplify the process of bundling items.
Preparation for Use	N/A
Diagrams (drawings, pictures)	
Steps for Use of Product	<ul style="list-style-type: none"> • Place items on Pro-Ties™ • Then lock one end under the other to secure bundle
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A

Reassembly Instructions	N/A
Packaging	Can be used within instrument containers, bags, wraps and pouches.
Sterilization	<p>Compatible with steam sterilization.</p> <ul style="list-style-type: none"> • Gravity steam autoclave at 121 °C (250 °F) for 30 minutes. • Recommended cycle in a pre-vac sterilizer. <ul style="list-style-type: none"> ○ 132 °C (270 °F) for 4 minutes ○ 135 °C (275 °) for 3 minutes in a pre-vac sterilizer. • Maximum extended cycle time 30 minutes 135 °C (275 °F). • Compatible with EO Sterilization.
Storage:	N/A
Additional Information	N/A

Related Healthmark Products	Clamps & Spreaders, Adjustable Stringer, Stringers
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	<p>Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com</p>