



Instructions for Use: Sauder Instrument Holder

Brand Name of Product	Sauder Instrument Holder
Generic Name of Product	Stringer
Product Code Number(s)	10-1780
Intended Use	Keeps ring handled instruments maximally exposed during sterilization.
Range of Applications for Product	Used for ring handled instruments.
Key Specifications of Product	<ul style="list-style-type: none"> Manufactured of stainless steel

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product

Description of Use (s)	Keeps ring handled instruments maximally exposed during sterilization
-------------------------------	---

Preparation for Use	
----------------------------	--

Diagrams (drawings, pictures)	 <p style="text-align: center;">Figure 1</p>  <p style="text-align: center;">Figure 2</p>  <p style="text-align: center;">Figure 3</p>
--------------------------------------	--

Steps for Use of Product	<ol style="list-style-type: none"> Unhook both bars of the stringer. Fig. 1 Place the ring handled instruments in the open position. The rings of the instrument are put through both bars of the stringer. Fig. 2 When finished placing the instruments through the bars, push the bars back underneath the hooks to secure them in place. Fig. 3 The instruments should be facing in a forward fashion.
---------------------------------	---

Interpretation of Test Results	
---------------------------------------	--

Contraindications of Test Results	
--	--

Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	<ul style="list-style-type: none"> • Before initial use of the stringer, cleaning is required. Manually wash stringer(s) with a neutral pH detergent and warm water.
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	<ul style="list-style-type: none"> • See instructions under “Steps for Use of Product” for directions on how to string instruments. • When using for sterilization of instruments, open to the optimal width for use in the instrument set. Consult the instrument manufacturer’s IFU for specific instructions. • Stringer are materially compatible with steam and EO sterilization. • Steam sterilization at 270°F (132°C) for 4 minutes or 275°F (135°C) for 3 minutes.
Storage	
Additional Information	
Related Healthmark Products	Stringers
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com