



Instructions for Use: Stainless Steel Washing Sleeve

Brand Name of Product	Stainless Steel Washing Sleeve
Generic Name of Product	Stainless Steel Washing Sleeve
Product Code Number(s)	07433, 07436, 07439, 07441
Intended Use	Intended for effective transportation, washing and storage of delicate items
Range of Applications for Product	
Key Specifications of Product	<ul style="list-style-type: none"> • Perforated Stainless Steel • 2 silicone caps

Shipping & Storage

Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product

Description of Use (s)	Manufactured for effective transportation, washing and storage of delicate items.
Preparation for Use	
Diagrams (drawings, pictures)	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place instrument inside washing sleeve 2. Secure caps to each end of the washing sleeve 3. Begin reprocessing in accordance to the item manufacturer’s IFU.
Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions

Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	Use cleaning solution approved for use with stainless steel.
Cleaning – Automated	<ul style="list-style-type: none"> • Select the “Instrument Cycle” on the automated washer. • Use cleaning solution approved for use with stainless steel.
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	
Packaging	
Sterilization	Steam Sterilization: <ul style="list-style-type: none"> • 135 °C / 275 °F / 3 min • 132 °C / 270 °F / 4 min
Storage	
Additional Information	

Related Healthmark Products	
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

