

Instructions for Use: Stainless Steel Washing Sleeve

Brand Name of Product	Stainless Steel Washing Sleeve
Generic Name of Product	Stainless Steel Washing Sleeve
Product Code Number(s)	SSWS-01, SSWS-02, SSWS-03, SSWS-04
Intended Use	Intended for effective transportation, washing, and storage of delicate items.
Range of Applications for Product	N/A
Key Specifications of Product	<ul style="list-style-type: none"> Perforated Stainless Steel Two (2) silicone caps.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Manufactured for effective transportation, washing and storage of delicate items.
Preparation for Use	N/A
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> Place instrument inside washing sleeve. Secure caps to each end of the washing sleeve. Begin reprocessing the (item/instrument) in accordance with the manufacturer's IFU.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	Use cleaning solution approved for use with stainless steel.
Cleaning – Automated	<ul style="list-style-type: none"> Select the “Instrument Cycle” on the automated washer. Use a cleaning solution approved for use with stainless steel.
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	Steam Sterilization at: <ul style="list-style-type: none"> 132 °C (270 °F) for 4 minutes 135 °C (275 °F) for 3 minutes.
Storage	N/A
Additional Information	N/A

Related Healthmark Products	
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc 18600 Malyn Blvd. Fraser, MI 48026

	1-586-774-7600 healthmark@hmark.com hmark.com
--	---