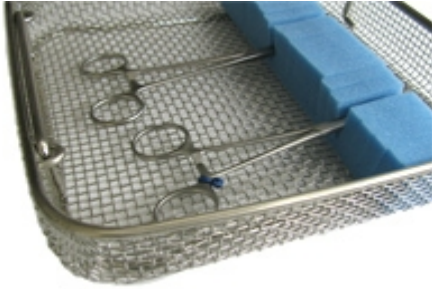

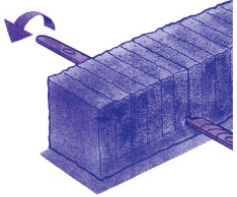




**Instructions for Use: Tray Anchors**

|  |  |
|--|--|
| <b>Brand Name of Product</b>             | Tray Anchors   |
| <b>Generic Name of Product</b>           | Tray Anchors   |
| <b>Product Code Number(s)</b>            | FTA-201, FTA-204   |
| <b>Purpose of Product</b>                | To secure instruments in a wide variety of instrument trays. Each anchor measure 18” long and can be easily trimmed to exact length required.  |
| <b>Range of Applications for Product</b> | Versatile and effective foam tray anchors are designed to secure a wide variety of instruments trays.  |
| <b>Key Specifications of Product</b>     | <ul style="list-style-type: none"> <li>• Manufactured of open cell, reticulated foam.</li> <li>• Slotted for easy placement of instruments.</li> <li>• Perforated for quick removal of individual sections to accommodate wider instruments.</li> <li>• May be cut to the desired width to fit in various size trays.</li> <li>• Single Use.</li> <li>• Compatible with steam and EO sterilization.</li> </ul> |

| <b>Shipping &amp; Storage</b>                 |  |
|---|--|
| <b>Shipping Conditions &amp; Requirements</b> |  |
| <b>Storage Conditions</b>                     |  |
| <b>Packaging Contents</b>                     | <ul style="list-style-type: none"> <li>• FTA-201: Pack of 16</li> <li>• FTA-204: Pack of 64</li> </ul> |
| <b>Shelf Life</b>                             |  |

| <b>Instructions for Using Product</b>    |  |
|--|--|
| <b>Description of Use (s)</b>            | Secure instruments between pre-cut sections – locking them in place with a twist under the profile cut. For a wider instrument, quickly remove one or more sections to create additional room.   |
| <b>Preparation for Use</b>               | <ol style="list-style-type: none"> <li>1. Trim foam anchor slightly larger than the tray to ensure a tight fit.</li> <li>2. An absorbent tray liner may be used to line the tray and the anchors positioned as required.</li> <li>3. The salvage edges from the pack can be cut and used for extra protection or rests.</li> </ol> |
| <b>Diagrams (drawings, pictures)</b>     |      |
| <b>Steps for Use of Product</b>          | <ol style="list-style-type: none"> <li>1. Twist and lock feature holds the instrument into place.</li> <li>2. Sections can be removed to allow for larger instruments.</li> </ol>  |
| <b>Interpretation of Results</b>         |  |
| <b>Contraindications of Test Results</b> |  |
| <b>Documentation</b>                     |  |
| <b>Special Warnings and Cautions</b>     |  |
| <b>Disposal</b>                          |  |

| <b>Reprocessing Instructions</b>       |  |
|--|--|
| <b>Point of use</b>                    |  |
| <b>Preparation for decontamination</b> |  |
| <b>Disassembly Instructions</b>        |  |
| <b>Cleaning – Manual</b>               |  |
| <b>Cleaning – Automated</b>            |  |
| <b>Disinfection</b>                    |  |

|   |   |
|---|---|
| <b>Drying</b>                               |   |
| <b>Maintenance, Inspection, and Testing</b> |   |
| <b>Reassembly Instructions</b>              |   |
| <b>Packaging</b>                            | See “ <b>Steps for Use of Product</b> ” section above.  |
| <b>Sterilization</b>                        | <ul style="list-style-type: none"> <li>● Steam sterilization at 270°F (132°C) for 4 minutes or 275°F (135°C) for 3 minutes.</li> <li>● Maximum extended time for 30 minutes at 275°F (135°C).</li> <li>● Compatible with EO sterilization.</li> </ul> |
| <b>Storage</b>                              |   |
| <b>Additional Information</b>               |   |

|  |   |
|--|---|
| <b>Related Healthmark Products</b>     |   |
| <b>Other Product Support Documents</b> | Prosys Brochure, Prosys Price List  |
| <b>Reference Documents</b>             |   |
| <b>Customer Service Contact</b>        | Healthmark Industries Company, Inc<br>18600 Malyn Blvd.<br>Fraser, MI 48026<br>1-586-774-7600<br><a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a><br><a href="http://hmark.com">hmark.com</a> |