



Instructions for Use: Window Style Tip Protectors

Brand Name of Product	Window Style Tip Protectors
Generic Name of Product	Tip Protectors
Product Code Number(s)	15.100.055, WSTP-001
Purpose of Product	Designed to protect tips during sterilization, transport, and storage.
Range of Applications for Product	For variety of small size items.
Key Specifications of Product	<ul style="list-style-type: none"> • Single use • Paper: 250 g wood free allowing excellent performance. • Pouch: Transparent pouch made of polyethylene-polypropylene laminate • Ink: Non-toxic water-solvent based ink (pH in between 9.0 and 9.5 at 25 °C)

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	<ul style="list-style-type: none"> • Temperature: 10 °C - 40 °C • Relative humidity: 20% < RH < 80% • Keep away from direct sunlight/keep dry
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	To protect tips during sterilization, transport, and storage.
Preparation for Use	N/A
Diagrams (drawings, pictures)	<p>Item Code- 15.100.055</p> <div style="display: flex; flex-wrap: wrap;"> <div style="width: 33%; text-align: center;">  <p>Figure 1</p> </div> <div style="width: 33%; text-align: center;">  <p>Figure 2</p> </div> <div style="width: 33%; text-align: center;">  <p>Figure 3</p> </div> <div style="width: 33%; text-align: center;">  <p>Figure 4</p> </div> <div style="width: 33%; text-align: center;">  <p>Figure 5</p> </div> </div> <p>Item Code- WSTP-001</p>

	   
	<p style="text-align: center;">Figure 1</p> <p style="text-align: center;">Figure 2</p> <p style="text-align: center;">Figure 3</p> <p style="text-align: center;">Figure 4</p>
Steps for Use of Product	<p>Item Code- 15.100.055</p> <ol style="list-style-type: none"> 1. Place the tip through top slotted insert (Fig 1 & Fig 2) 2. Continue sliding the item down the backside of the protector 3. Tip should emerge through the second slotted insert in front of the protector (Fig 3) 4. Make sure tip slides inside of transparent window pouch (Fig 4) 5. Keep sliding item down until it is secure. (Fig 5). <p>Item Code- WSTP-001</p> <ol style="list-style-type: none"> 1. Place the tip through both slots (Fig 1) 2. Continue sliding the item down the backside of the protector 3. Make sure tip slides inside of transparent window pouch (Fig 2) 4. Keep sliding the item down until it is secure 5. Fold the flap over and secure underneath the top slot to ensure the item is kept open. (Fig 3, Fig 4).
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	Single use; dispose after use in accordance with facility policy.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	<ul style="list-style-type: none"> • Compatible with steam sterilization. Recommended cycle is 4 minutes at 132 °C or 3 minutes at 135 °C in a pre-vacuum sterilizer • Material Compatibility <ul style="list-style-type: none"> ○ Steam up to 145 °C ○ EO
Storage	N/A
Additional Information	N/A

Related Healthmark Products	Transparent Tip Protectors, Vented Instrument Guards, Duo-Vented Guards, Cylindrical Style Instrument Guards
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd.

Fraser, MI 48026
1-586-774-7600
healthmark@hmark.com
hmark.com