

Instructions for use

Instruments Guards

Includes all products except for Web Guards #09-9-001 through 09-9-007

Intended Use:

The Instrument Guards are intended to protect surgical instruments through handling and sterilization.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for Use:

- Place guard over clean instrument.
- Guards may be left on during handling, storage, and sterilization by steam or ethylene oxide.
- Immediate Use steam sterilization (IUSS) for 4 minutes at 132° C (270° F).
- Gravity steam sterilization for 30 minutes at 121° C (250° F).

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

 **WARNING:** This product can expose you to Diisononyl phthalate (DINP), which is known to the State of California to cause cancer. For more information go to www.P65Warnings.ca.gov.