

Instructions for Use: Tracheal Cleaning Brush

Brand Name of Product	Tracheal Cleaning Brush
Generic Name of Product	Cleaning Brushes
Product Code Number(s)	SUCT01-10, SUCT02-10, SUCT03-10, SUCT04-10, SUCTPANA-40
Intended Use	Intended for cleaning tracheal channels.
Range of Applications for Product	Tracheal channels of appropriate diameter and length.
Key Specifications of Product	<ul style="list-style-type: none"> • Thermoplastic fibers • Encapsulated within a twisted-curved, stainless-steel frame. • Flexible thermoplastic handle/wand

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	Case of 40
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Designed for internal cleaning.
Preparation for Use	<ul style="list-style-type: none"> • Cleaning any cannulated item requires the use of the proper size cleaning brush in accordance with the cannulated item manufacturer (Mfr.)'s IFU. • It is important/critical to: <ul style="list-style-type: none"> ○ Match the diameter size of the brush you are using with the channel you are cleaning. ○ Select a brush longer than the channel to ensure the entire channel receives cleaning action. ○ Ensure the material composition of the brush bristles is suitable for the item to be cleaned. This should be in compliance with the cannulated item manufacturer's IFU. • The Healthmark brush-sizing chart, or similar type of tool, may be used to select the proper diameter brush. • Inspect brushes, making sure there is no fraying or damage. If you note any discrepancies, discard the brush and do not use. • Brushes should also be inspected between uses for any residual organic soil that may cross-contaminate the next item cleaned. If there is visual soil, the brush should be disposed of or cleaned in accordance with the method(s) described below.
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ul style="list-style-type: none"> • Consult the Mfr.'s IFU for the item you are cleaning and follow those instructions closely. • Insert the cleaning brush by the non-brush end in the proximal aperture, pushing the shaft into the instrument channel, until it reappears at the end.
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • Check the integrity and the conditions of the brush and stem prior to using. • Do not use for cleaning plastic and/or fragile materials, such as silicone. • Avoid brush compression during sterilization and storage. • Protect yourself from brush splatter/contamination by donning appropriate PPE. • Inspect for visible soil and clean between each instrument to limit chance for cross contamination. • Discard the disposable tips after use.
Disposal	Single-use.

Reprocessing Instructions

Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	Instrument Care Brochure, Instrument Care Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com