



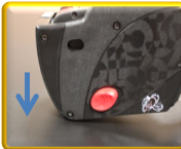






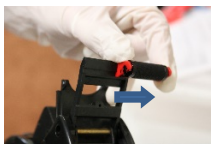

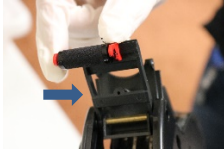


Instructions for Use: Handheld labeling System

Brand Name of Product	Handheld Labeling System
Generic Name of Product	Label Gun and labels
Product Code Number(s)	H-362026, H-369020, H369021, H369023, H369024, H369010, H369012, H369013, H369014, H369020S, H369021S, H369023S, H369024S, H369010S, H369012S, H369013S, H369014S.
Purpose of Product	Hand-held Labeling System has been designed specifically with Central Sterile in mind for the labeling of any items that you need to be autoclaved - from sterile linen packs to instrument trays, and peel pouches.
Range of Applications for Product	Great versatility for labeling in other hospital departments such as dietary and pharmacy.
Key Specifications of Product	All of the labels are compatible with Steam and EO sterilization. The labels with an “S” at the end of the product code number are compatible with Gas-Plasma Sterilization.

Shipping & Storage	
Shipping Conditions & Requirements	
Storage Conditions	
Packaging Contents	
Shelf Life	

Instructions for Using Product	
Description of Use (s)	Label Gun is used for labeling a variety of products and packages being autoclaved.
Preparation for Use	
Diagrams (drawings, pictures)	<p>Setting the characters</p>   <p>1 2</p> <p>Labeling</p>     <p>1 2 3 4</p> <p>Replacing the label roll</p>    <p>1 2 3</p>  

	4	5	
Replacing the ink-roll			
			
1A	1B	2A	
			
2B	3		

Steps for Use of Product	<p>Setting the characters</p> <ol style="list-style-type: none"> 1. Pull adjustment knob to desired position and turn knob to select character to print. 2. Press knob in before printing (Important: failure to do so before operating could cause damage). <p>Labeling</p> <ol style="list-style-type: none"> 1. Place your hand around the handle and squeeze you fingers to your palm. 2. Release the handle back to the starting position. 3. Pulling the handle will release the label from the opposite end of the label gun. 4. Repeat to use another label on another product or package. <p>Replacing the label-roll</p> <ol style="list-style-type: none"> 1. Press and slide red release for the top slide back. The release is on both sides of the label gun. 2. Pull out the chamber in order to insert the roll of labels. 3. Place new label roll inside with the label flap coming over the top. 4. Secure the end of the label roll to the top end of the label gun. 5. When in use, the label will dispense through the chamber. <p>Replacing the ink-roll</p> <ol style="list-style-type: none"> 1. Lift-up the ink roll cover at the front of the label gun. 2. Remove the ink roller cartridge by sliding it out from the groove on either side. (We recommend wearing gloves. Your hands will get ink on them.) 3. Replace with new ink roller Prix by sliding cartridge back into the groove and close the cover.
---------------------------------	---

Interpretation of Results	
Contraindications of Test Results	
Documentation	
Special Warnings and Cautions	
Disposal	

Reprocessing Instructions	
Point of Use	
Preparation for Decontamination	
Disassembly Instructions	
Cleaning – Manual	Use damp cloth to wipe down the Label Gun.
Cleaning – Automated	
Disinfection	
Drying	
Maintenance, Inspection, and Testing	
Reassembly Instructions	

Packaging	
Sterilization	All the labels are compatible with Steam and EO sterilization. The labels with an “S” at the end of the product code number are compatible with Gas-Plasma Sterilization. (DO NOT PLACE THE LABEL GUN IN ANY STERILIZER).
Storage	
Additional Information	

Related Healthmark Products	Premium Label Gun and two-part labels
Other Product Support Documents	Labeling Brochure, Labeling Price List
Reference Documents	
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com