


## Instructions for Use: Hanging Tag

<b>Brand Name of Product</b>	Hanging Tag
<b>Generic Name of Product</b>	Labels
<b>Product Code Number(s)</b>	HT-001
<b>Purpose of Product</b>	For identifying trays, containers, and other key assets.
<b>Range of Applications for Product</b>	The surface of the tag is ideal for barcode labels.
<b>Key Specifications of Product</b>	<ul style="list-style-type: none"> <li>• Composition: Paper, 216 g/m<sup>2</sup></li> <li>• Color: White</li> <li>• Size: 3.5- x 1.9- x 0.78 inches</li> <li>• Non-adhesive and heat resistant up to 135 °C (275 °F).</li> </ul>

### Shipping & Storage

<b>Shipping Conditions &amp; Requirements</b>	N/A
<b>Storage Conditions</b>	N/A
<b>Packaging Contents</b>	Case of 1,000.
<b>Shelf Life</b>	N/A

### Instructions for Using Product

<b>Description of Use(s)</b>	Used to identify trays, containers, and other key assets.
<b>Preparation for Use</b>	N/A
<b>Diagrams (drawings, pictures)</b>	
<b>Steps for Use of Product</b>	<ol style="list-style-type: none"> <li>1. Tag items being identified in the tray or container.</li> <li>2. Secure tag by tying the precut slots to either the tray, container, or other equipment. (Note: For security, use a lock when using closed containers.)</li> </ol>
<b>Interpretation of Results</b>	N/A
<b>Contraindications of Test Results</b>	N/A
<b>Documentation</b>	N/A
<b>Special Warnings and Cautions</b>	Single use.
<b>Disposal</b>	N/A

### Reprocessing Instructions

<b>Point of Use</b>	N/A
<b>Preparation for Decontamination</b>	N/A
<b>Disassembly Instructions</b>	N/A
<b>Cleaning – Manual</b>	N/A
<b>Cleaning – Automated</b>	N/A
<b>Disinfection</b>	N/A
<b>Drying</b>	N/A
<b>Maintenance, Inspection, and Testing</b>	N/A
<b>Reassembly Instructions</b>	N/A
<b>Packaging</b>	N/A

<b>Sterilization</b>	<ul style="list-style-type: none"> <li>• Steam sterilization up to: <ul style="list-style-type: none"> <li>○ 135 °C (275 °F) for three (3) minutes.</li> <li>○ 132 °C (270 °F) for four (4) minutes.</li> </ul> </li> <li>• Materially compatible with EO sterilization.</li> </ul>
<b>Storage</b>	N/A
<b>Additional Information</b>	N/A

<b>Related Healthmark Products</b>	N/A
<b>Other Product Support Documents</b>	Label Price List, Label Brochure
<b>Reference Documents</b>	N/A
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com