
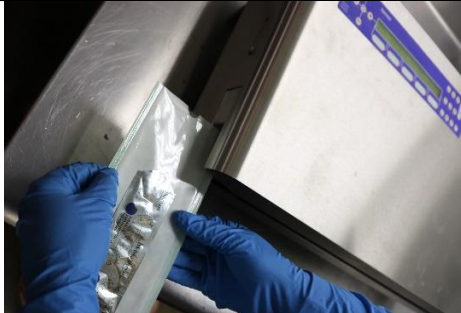
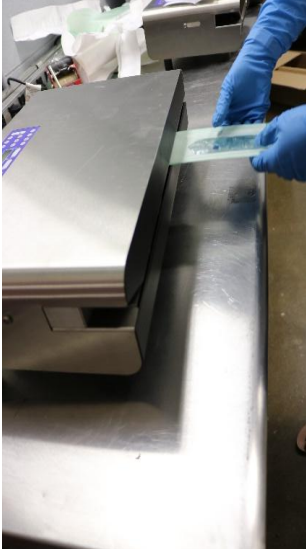


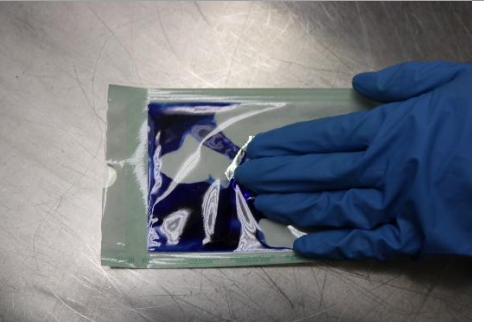
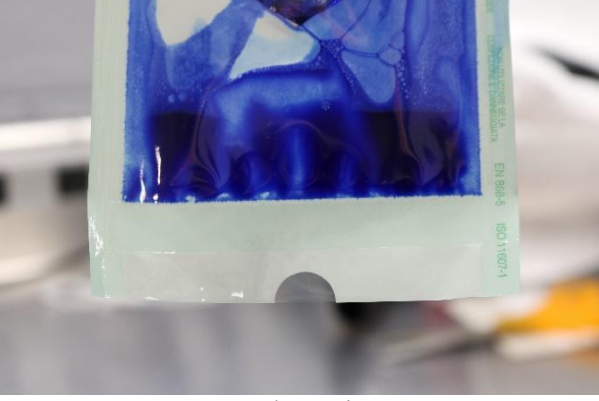




Brand Name of Product	Pouch Seal Integrity Test
Generic Name of Product	Pouch Seal Integrity Test
Product Code Number(s)	PS-001, TPS-001
Intended Use	To test the integrity of the seals for heat seal and self-seal pouches.
Range of Applications for Product	Testing the sealing process for both paper/poly and Tyvek®/poly packaging material for an intact seal.
Key Specifications of Product	<ul style="list-style-type: none"> • PS-001: paper/plastic pouches-Blue dye • TPS-001: Tyvek® pouches-Red dye • Push Indicator Dye Test® • Seal Test Results Sheets

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Testing the seal integrity with an indicator dye test.
Preparation for Use	Put on gloves to prevent any ink that might leak out of the pouch on to the hands.
Diagrams (drawings, pictures)	 <p style="text-align: center;">Figure 1</p>  <p style="text-align: center;">Figure 2</p>  <p style="text-align: center;">Figure 2a</p>  <p style="text-align: center;">Figure 3</p>

		
	Figure 3a	Figure 3b
		
	Figure 4	
Steps for Use of Product	<ol style="list-style-type: none"> 1. Test daily for each sealing device. 2. With gloves on, place the Push Indicator Dye Test[®] in the pouch and seal pouch. (Fig. 1, 2, 2a). 3. Once sealed, place pouch on wipeable surface as the ink will soak through the paper on the back of the pouch. 4. Break the test pack by sliding three fingers in the direction of the arrow of the push indicator to break so the ink drains out and into the pouch towards the seal. (Fig. 3, 3a, 3b). 5. Once the ink has been released from the packet, allow the dye to be in contact along the full length of the seal. (Fig. 4) 6. Visually examine immediately. Check for broken areas or channels that have occurred in the heat seal joint. 7. Document the results and approve test results on the Seal Test Results Sheet. 	
Contraindications of Test Results	N/A	
Documentation	Record test results on log sheet located at www.hmark.com	
Special Warnings and Cautions	N/A	
Disposal	N/A	

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	Sterilization Brochure, Sterilization Price List

Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com