

Instructions for Use: EndoPro-Cam® Adapter for Stryker Rigid Scopes

Brand Name of Product	EndoPro-Cam® Adapter for Stryker Rigid Scopes
Generic Name of Product	Adapter for Stryker Rigid Scopes
Product Code Number(s)	EPRO-STK
Intended Use	An adapter ring to attach the Stryker Rigid Scopes between the lens and camera on the EndoPro-Cam® to allow enough distance for better visual inspection of the rigid scope.
Range of Applications for Product	An adapter ring for Stryker Rigid Scopes that attaches between the lens and camera.
Key Specifications of Product	<ul style="list-style-type: none"> • Dimensions: 3.78- x 2.4- x 0.43 inches • Diameter: <ul style="list-style-type: none"> ○ External: 31 mm ○ Internal: 24.5 mm • Adapter ring: <ul style="list-style-type: none"> ○ 5 mm ○ Converts C-Mount to CS-Mount lens for use on a CS-Mount camera. • Weight: 0.634 ounces • Material: Matte-aluminum

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A





Instructions for Using Product	
Description of Use(s)	An adapter ring for Stryker Rigid Scope that attaches between the lens and camera.
Preparation for Use	N/A
Diagrams (drawings, pictures)	<div style="display: flex; flex-wrap: wrap;"> <div style="width: 50%; text-align: center;">  <p>Figure 1</p> </div> <div style="width: 50%; text-align: center;">  <p>Figure 1a</p> </div> <div style="width: 50%; text-align: center;">  <p>Figure 2</p> </div> <div style="width: 50%; text-align: center;">  <p>Figure 2a</p> </div> </div>






Figure 3



Figure 3a



Figure 4

	  
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place the adapter ring over the camera of the EndoPro-Cam®. Screw on the adapter ring by turning the ring to the right, so it is secured to the camera. (Fig. 1, 1a). Note: Do not over tighten as this will cause difficulty to loosen when trying to remove the adapter ring from the camera. 2. Place the Stryker Rigid Scope over the adapter ring. Screw on the Stryker Rigid Scope to adapter ring by turning the scope to the right, so it is secured to the adapter ring. (Fig. 2, 2a). 3. Place the light adapter on the light source on the Stryker Rigid Scope. Screw on the light adapter by turning to the right to tighten, so it is secured to the light source. (Fig. 3, 3a).
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	Remember to remove the adapter ring from the Stryker Rigid Scope before sending to the OR for use. (Fig. 4) .
Disposal	N/A

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	Unscrew adapter ring counterclockwise from the back of the EndoPro-Cam® before cleaning and disinfection.
Cleaning – Manual	<ol style="list-style-type: none"> 1. Pay particular attention to the edges of the adapter ring. 2. Using a fresh, 70% Isopropyl Alcohol (IPA) pre-moistened wipe to thoroughly wipe the adapter ring. Follow the IFU for the wipe for recommended wipe technique and contact time. 3. Visually inspect for any residual soil. 4. Place clean adapter ring in an appropriate dry storage area.
Cleaning – Automated	N/A
Disinfection	Disinfect with Isopropyl wipes.
Drying	Manually dry with a non-linting wipe.
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	<ol style="list-style-type: none"> 1. Reattach adapter ring to back of EndoPro-Cam®. 2. Secure by turning the adapter ring clockwise. (Note: Do not over tighten adapter ring.)
Packaging	N/A

Sterilization	N/A
Storage	N/A
Additional Information	N/A
Related Healthmark Products	EndoPro-Cam®
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com

2022 Suzanne Latta