

Brand Name of Product	SU Micro & SU L														
Generic Name of Product	Single-use Brushes														
Product Code Number(s)	055-1824-50, SUL05-50, SUL07-50, SUL09-50, SUL12-50, SUL16-50														
Intended Use	A tool for cleaning internal lumens/channels														
Range of Applications for Product	Cleaning internal lumens/channels														
Key Specifications of Product	<p>Single-use. Available in a variety of sizes:</p> <table border="1"> <thead> <tr> <th>Product Codes</th> <th>Key Specifications of Product</th> </tr> </thead> <tbody> <tr> <td>055-184-50</td> <td>3 mm Dia. x 60 cm L</td> </tr> <tr> <td>SUL05-50</td> <td>5.5 mm Dia. x 60 cm L</td> </tr> <tr> <td>SUL07-50</td> <td>8 mm Dia. x 60 cm L</td> </tr> <tr> <td>SUL09-50</td> <td>11 mm Dia. x 60 cm L</td> </tr> <tr> <td>SUL12-50</td> <td>13 mm Dia. x 60 cm L</td> </tr> <tr> <td>SUL16-50</td> <td>18 mm Dia. x 60 cm L</td> </tr> </tbody> </table>	Product Codes	Key Specifications of Product	055-184-50	3 mm Dia. x 60 cm L	SUL05-50	5.5 mm Dia. x 60 cm L	SUL07-50	8 mm Dia. x 60 cm L	SUL09-50	11 mm Dia. x 60 cm L	SUL12-50	13 mm Dia. x 60 cm L	SUL16-50	18 mm Dia. x 60 cm L
Product Codes	Key Specifications of Product														
055-184-50	3 mm Dia. x 60 cm L														
SUL05-50	5.5 mm Dia. x 60 cm L														
SUL07-50	8 mm Dia. x 60 cm L														
SUL09-50	11 mm Dia. x 60 cm L														
SUL12-50	13 mm Dia. x 60 cm L														
SUL16-50	18 mm Dia. x 60 cm L														

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	N/A
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Intended as a tool for the cleaning of internal lumens/channels.
Preparation for Use	<ul style="list-style-type: none"> ● Cleaning a cannulated item requires the proper size brush in accordance with the manufacturer's (Mfr.'s) IFU. ● The Healthmark Brush Sizing Chart, or similar type of tool, may be used to select the proper diameter of the brush. It is important that the brush's: <ul style="list-style-type: none"> ○ Diameter should be slightly greater than the channel to be cleaned to ensure effective contact with surfaces. ○ Length is longer than the channel to ensure the entire channel receives cleaning action. ● (Prior to each use), inspect brushes to be certain they are not a) damaged, b) have loose, missing, or frayed bristles, or c) other signs of damage. If discrepancies are noted, do not use the brush and discard. ● Ensure the brush's material composition is suitable for the item to be cleaned and in compliance with its Mfr.'s IFU. ● For lumens open on both ends all brush bristles must exit the distal tip of the item being cleaned. ● Brushes should be inspected and cleaned in between each passthrough of an item if passed through more than once. <ul style="list-style-type: none"> ○ Check for and clean any remaining bioburden or debris on or in the bristles (even if there is none, clean the brush anyway) with fingertips or second brush for stubborn soils ○ Inspect the brush and shaft to ensure no damage (i.e., broken or matted bristles or kinked or damaged shafts) has occurred during the previous pass through. If so, dispose of the brush and use a new one to complete the cleaning process.
Diagrams (drawings, pictures)	
Steps for Use of Product	<p>Consult the Mfr.'s IFU of the item being cleaned.</p> <ol style="list-style-type: none"> 1. <i>Push Method:</i> Insert the cleaning brush into lumen using short strokes. Feed the brush through the lumen until it emerges from the distal end. 2. <i>Pull Method:</i> Insert the non-bristle end of the brush into the lumen, then follow with the bristles. (NOTE: This insertion technique optimizes cleaning action and prevents/reduces kinking or bending at the point where the brush is inserted.)

	<ol style="list-style-type: none"> 3. (Important!) Ensure the brush is long enough to exit the lumen/channel completely when brushing so that the bristles can be examined and cleaned before next use. 4. Twist the brush (helical spinning of the brush) while inserting it through the entire length of the lumen to help remove debris. 5. While brushing, the brush should pass freely through the lumen. <ol style="list-style-type: none"> a. If significant resistance is encountered, stop; pull back. b. (Caution.) Continuing to brush/force through the resistance might cause damage to the lumen. <p>Click Here to view the Brush Support White Paper PDF.</p>
Interpretation of Test Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	<ul style="list-style-type: none"> • Brushes are intended only as cleaning tools (<i>they are not for use in patient care</i>). • Wear proper PPE when cleaning contaminated items. • The brushes are intended for single-use. <ul style="list-style-type: none"> ○ A “use” should be defined by the healthcare facility upon a multi-disciplinary risk assessment. ○ A “use” might be each: <ul style="list-style-type: none"> ▪ Item. ▪ Set or case cart of instruments. ▪ Shift or day the brush is used.
Disposal	It is recommended to dispose of SU Micro & SU L in the biohazard container because tube brushes are utilized to clean patient-used items (contaminated).

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	N/A
Storage	N/A
Additional Information	N/A
Related Healthmark Products	Brush Cleaning Tools
Other Product Support Documents	ProSys™ Brochure, ProSys™ Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com