



Instructions for Use: Dual UnderGuard™ Dry Mats

Brand Name of Product	Dual UnderGuard™ Dry Mats
Generic Name of Product	Dry Mats
Product Code Number(s)	DM-2030PP, DM-2030GN, DM-1520PP, DM-1520GN, DM-1224PP, DM-1224GN, DM-1212PP
Purpose of Product	To be used under and inside of trays (with the color side facing down) to protect against handling. Simultaneously provides an absorbent inner layer for the dispersion of condensate.
Range of Applications for Product	Provides protection useful for heavy trays and baskets.
Key Specifications of Product	<ul style="list-style-type: none"> Dry mats are made from polypropylene and viscose.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	<ul style="list-style-type: none"> Temperature: 15 °C (59 °F) to 30 °C (86 °F) Humidity: 30–70% rh
Packaging Contents	N/A
Shelf Life	Five (5) years from the manufacturer’s date.

Instructions for Using Product	
Description of Use(s)	Strong, spun polypropylene sheet with an absorbent layer to protect against sharp-edged trays, and promotes the dispersion of condensate.
Preparation for Use	N/A
Diagrams (drawings, pictures)	
Steps for Use of Product	Liner may be placed either under the tray or inside the bottom of the tray.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	
Special Warnings and Cautions	Dry mats are not compatible with VH2O2 sterilization processes.
Disposal	Single use. Dispose of according to facility’s policy.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	<ul style="list-style-type: none"> Steam sterilization at: <ul style="list-style-type: none"> 132 °C (270 °F) for 4 minutes 135 °C (275 °F) for 3 minutes. Gravity steam sterilization at 121 °C (250 °F) for 30 minutes. Maximum tested time/temperature: 30 minutes at 135 °C (275 °F). Materially compatible with EO sterilization.
Storage	N/A
Additional Information	N/A

Related Healthmark Products	N/A
Other Product Support Documents	Sterilization Brochure, Sterilization Price List
Reference Documents	N/A

Customer Service Contact

Healthmark Industries Company, Inc.
Fraser, MI 48026
1-586-774-7600
healthmark@hmark.com
hmark.com