

Instructions for Use: OrthoShield™ Soaker Sheets

Brand Name of Product	OrthoShield™ Soaker Sheets
Generic Name of Product	Paper Soaker Sheets
Product Code Number(s)	SK-2030, SK-1520, SK-1224, SK-1010, SK-1626
Purpose of Product	Specifically developed for heavy-duty use. Most commonly under trays to protect wraps from sharp corners and uneven surfaces of trays and baskets, while also providing rapid absorption of condensate.
Range of Applications for Product	For heavy sets, which often present the greatest challenge to the integrity of packaging and are most prone to the occurrence of wet pack.
Key Specifications of Product	<ul style="list-style-type: none"> • Single use. • Non-linting. • Use of soaker sheet greatly facilitates the production of dry packs.

Shipping & Storage	
Shipping Conditions & Requirements	N/A
Storage Conditions	<ul style="list-style-type: none"> • Temperature: 15 °C (59 °F) to 30 °C (86 °F) • Humidity: 30–70% rh.
Packaging Contents	N/A
Shelf Life	N/A

Instructions for Using Product	
Description of Use(s)	Use appropriate size wrap for tray selected.
Preparation for Use	N/A
Diagrams (drawings, pictures)	N/A
Steps for Use of Product	<ol style="list-style-type: none"> 1. Place soaker sheet on top of wrap. 2. Center where tray will be placed. 3. Place tray on top of soaker sheet (centered). 4. Wrap tray following standard technique.
Interpretation of Results	N/A
Contraindications of Test Results	N/A
Documentation	N/A
Special Warnings and Cautions	N/A
Disposal	<ul style="list-style-type: none"> • Single use. • Dispose of in compliance with facility policy for disposing of all packaging.

Reprocessing Instructions	
Point of Use	N/A
Preparation for Decontamination	N/A
Disassembly Instructions	N/A
Cleaning – Manual	N/A
Cleaning – Automated	N/A
Disinfection	N/A
Drying	N/A
Maintenance, Inspection, and Testing	N/A
Reassembly Instructions	N/A
Packaging	N/A
Sterilization	<ul style="list-style-type: none"> • Steam sterilization at: <ul style="list-style-type: none"> ○ 132 °C (270 °F) for 4 minutes ○ 135 °C (275 °F) for 3 minutes. • Gravity steam sterilization at 121 °C (250 °F) for 30 minutes. • Maximum extended time at 135 °C (275 °F) for 30 minutes. • Compatible with EO sterilization.
Storage	N/A

Additional Information	N/A
Related Healthmark Products	N/A
Other Product Support Documents	Sterilization Brochure, Sterilization Price List
Reference Documents	N/A
Customer Service Contact	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 healthmark@hmark.com hmark.com